

# Headrests


## MULTI-AXIS OFFSET HEADREST HARDWARE

**Features:**

- Adjusts horizontally, vertically, and pivots on a ball
- Ball diameter is 1", rod is 6" long
- Features a positioning stop collar which allows you to remove the headrest assembly without changing the desired headrest position
- Mounting plate is compatible with both 1 7/8" and 2" on center hole spacing
- Easily attaches to power wheelchairs
- Use with any headrest pad on pages 36-39 (sold separately)

Description	Part Number
Multi-Axis Offset Headrest Hardware	50215

CE compliant

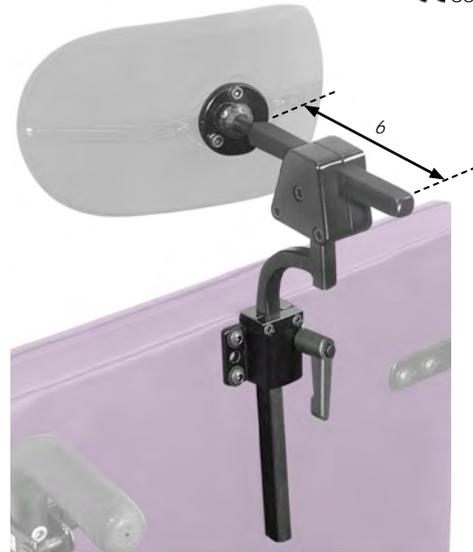

HEADREST PAD  
SOLD SEPARATELY

CUSTOM SEATING

BODY POSITIONING



*The Multi-Axis Hardware  
and Surelock Hardware  
both easily attach to  
power wheelchairs*

HEADREST PADS  
SOLD SEPARATELY



WHEELCHAIR TRAYS

ASSISTIVE TECHNOLOGY

PATIENT TRANSFER

## **NEW** HEADREST HORIZONTAL SLIDE MOUNT

**Features:**

- Specially modified channel allows the headrest mounting bracket to move laterally left or right approximately 2" off centerline
- Use with Therafin's Multi-Axis and Surelock headrest hardware
- Accommodates a user with asymmetrical alignment resulting in the head position not along the centerline of the back
- Outside dimensions: 6.75" wide x 1.9" tall

Description	Part Number
Horizontal Slide Mount channel and attaching bolts	32975




# Multi-Axis Offset Headrest Hardware

-  Caution: Read Instructions before operating device.
-  Caution: Device should be installed by a trained wheelchair technician.
-  Caution: Device should be attached to a wheelchair back that conforms with ISO16840-3 or equivalent.
-  Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
-  Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

## Product Identification

Components of the Multi-Axis Offset Headrest Device.

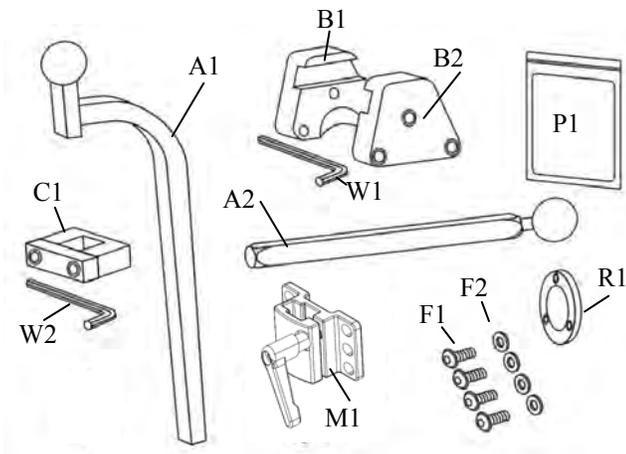


## Clean before assembly and use


Components may have a protective coating to shield from environmental damage. Use wipe (P1) to clean components prior to assembly and use.

## Installation and Assembly


1. Attach M1 to Wheelchair back using F1 and F2  
Tighten securely with wrench W2.

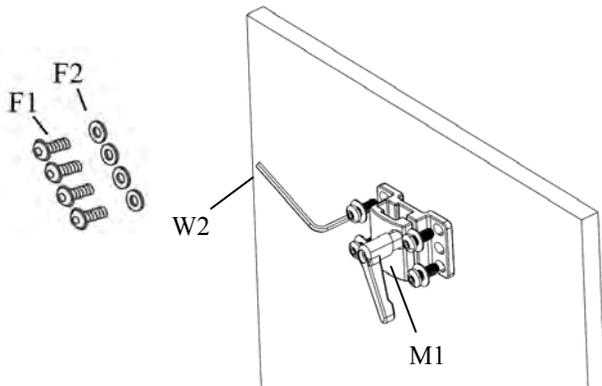


2. Place C1 onto A1 and insert A1 into M1. Tighten Knob.

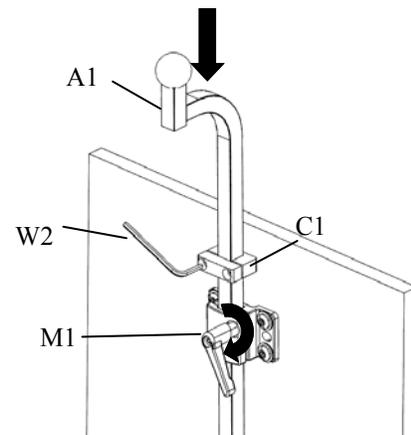

3. Place C1 in desired position and tighten bolts with wrench W2.

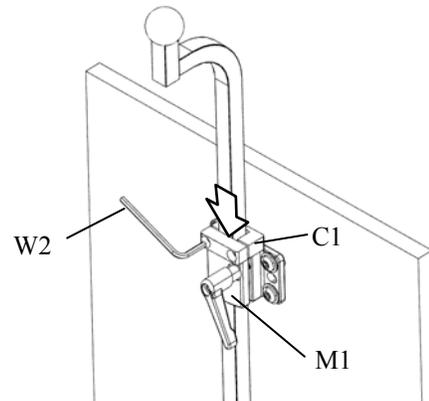


4. Assemble B1, B2, R1 and A2 as shown.  
Tighten Bolts with Wrench W1.

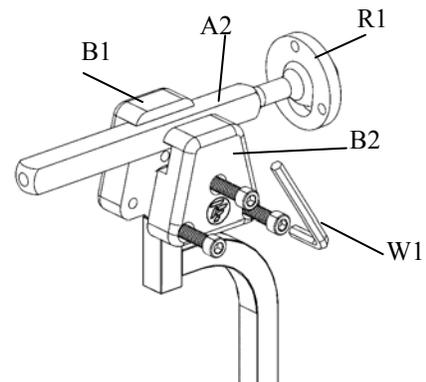

## Multi-Axis Offset Headrest Hardware


### 5. Attach headrest. (See Headrest Instructions)

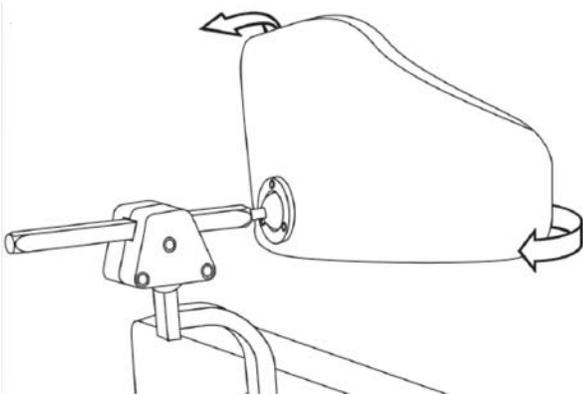

### Adjustment

-  Caution: Device should be installed by a trained wheelchair technician.
-  Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
-  Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

Adjustments shown below. Re-tighten all fasteners after desired position is obtained.

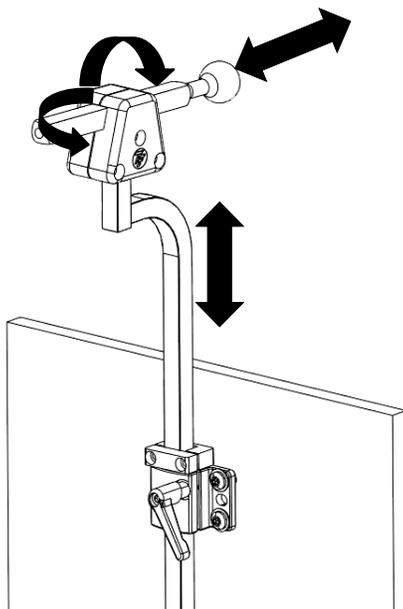

### Care and Cleaning

-  Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth.  
Air Dry.



2. Check all fasteners and retighten as needed.

### Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.



 Therafin Corporation  
9450 W Laraway Road  
Frankfort, IL 60432 USA  
800-843-7234  
[www.therafin.com](http://www.therafin.com)




**EMERGO EUROPE**  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands

# Headrest Horizontal Slide Mount


- ⚠ Caution: Read Instructions before operating device.
- ⚠ Caution: Device should be installed by a trained wheelchair technician.
- ⚠ Caution: Device should be attached to a wheelchair back that conforms with ISO16840-3 or equivalent.
- ⚠ Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
- ⚠ Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.
- ⚠ Caution: Device should be used with a headrest that conforms with ISO16840-3 or equivalent.

## Product Identification

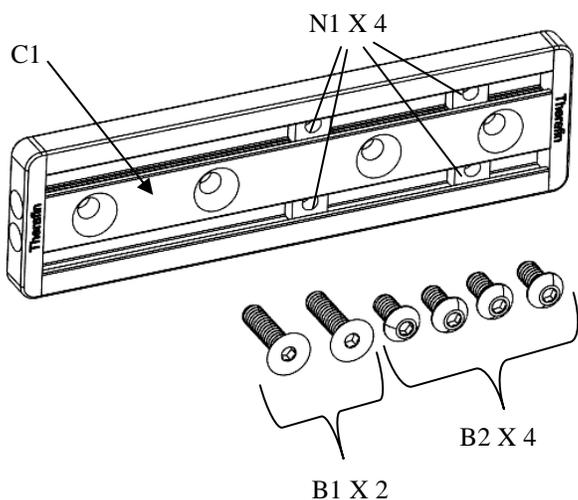


## To Install:

Image below shows installed device.

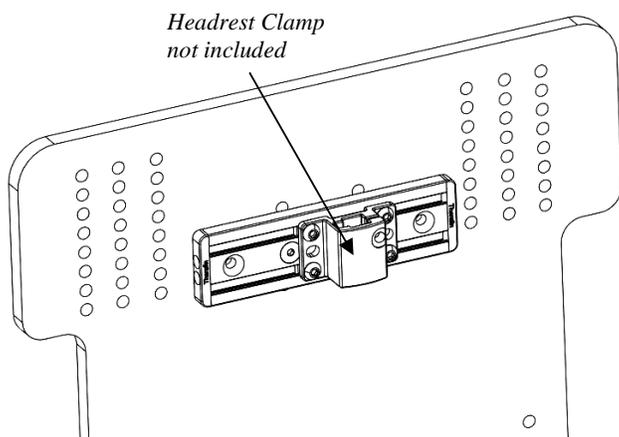


## To Install (continued):

**Step 1:** Align C1 with holes in backrest. Install using bolts B1 and tighten securely.

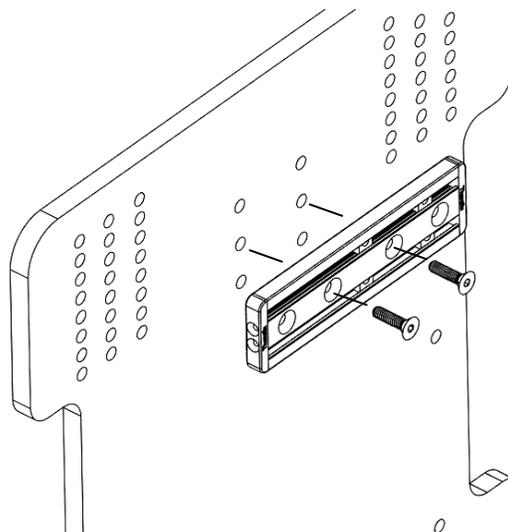


**Step 2:** Attach Headrest Clamp (not included) using bolts B2 threaded into nuts N1.

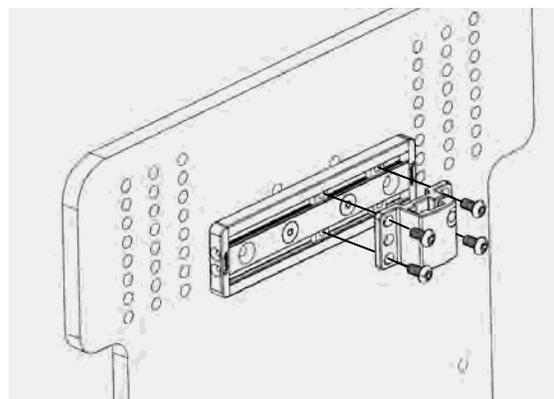

## To Adjust:

Loosen Bolts B2 and slide clamp along channel C1.

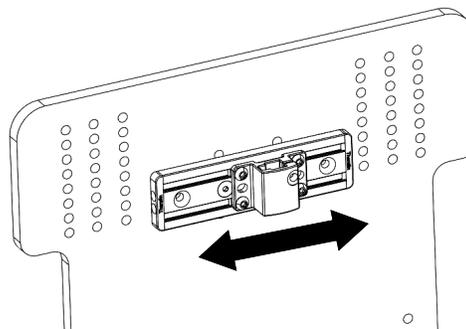

Therafin Corporation  
9450 W Laraway Road  
Frankfort, IL 60423 USA  
800-843-7234