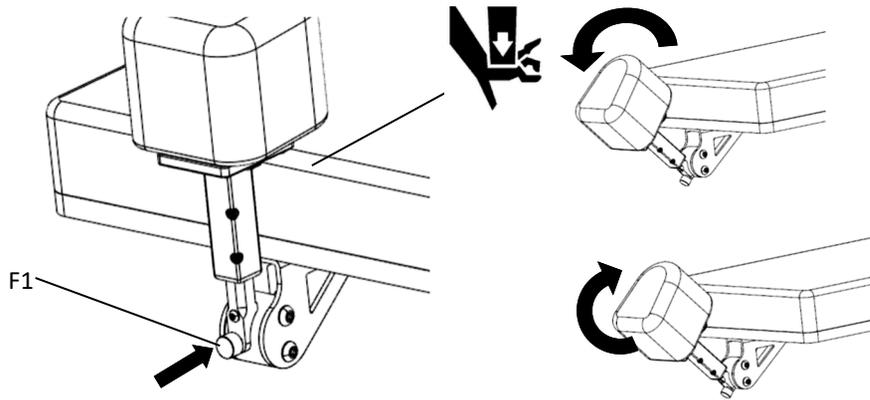
PARA OPERAR:

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

Flip Down Medial Thigh Support

English

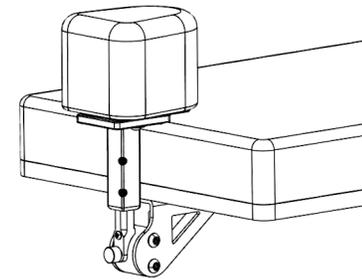Image below shows installed device.


Product Identification

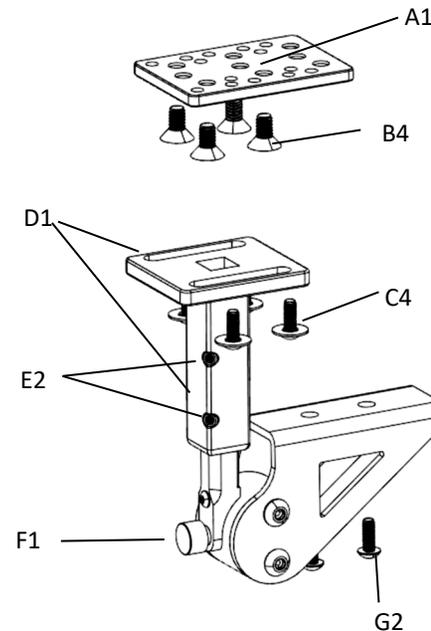
- ⚠ Caution: Read Instructions before operating device.
- ⚠ Caution: Device should be installed by a trained wheelchair technician.
- ⚠ Caution: Device should be attached to a wheelchair back that conforms with ISO16840-3 or equivalent.
- ⚠ Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
- ⚠ Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

To Install:

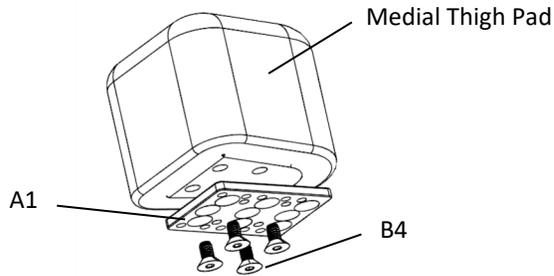
Step 1: Install plate (A1) on Medial Thigh Pad (not included) using bolts (B4). Tighten bolts securely

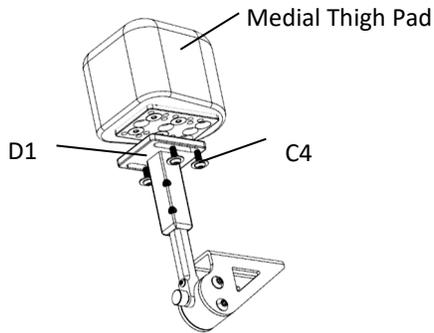Step 2: Attach Pad to Post (D1) using Bolts and washers (C4). Tighten bolts securely.

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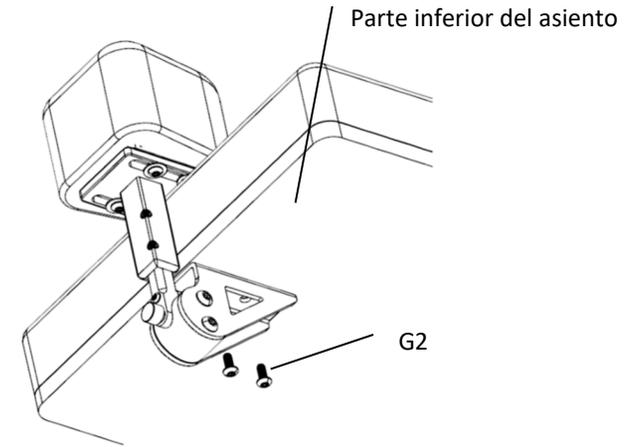
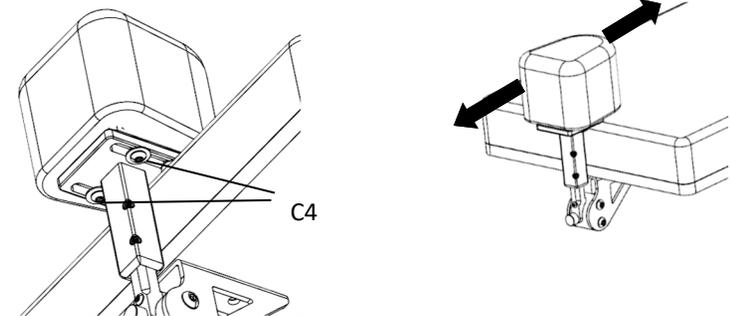
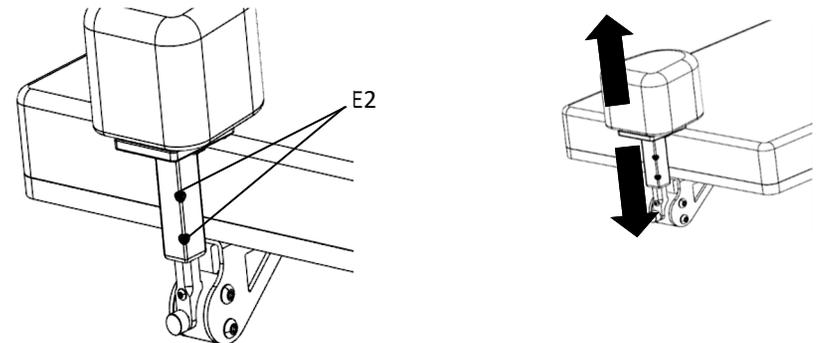
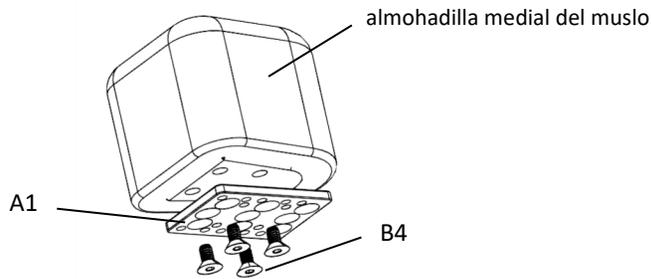
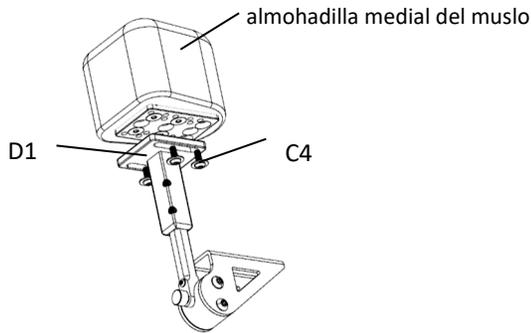
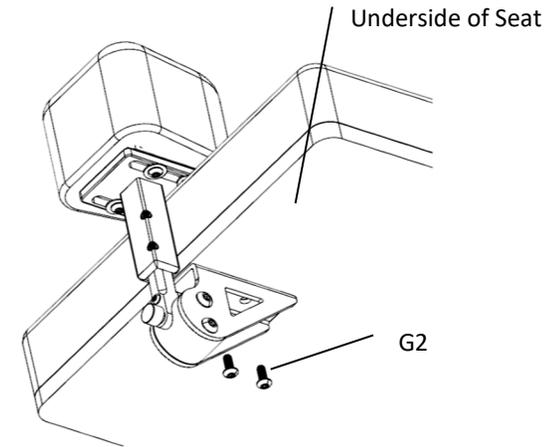
Step 3: Attach device to wheelchair seat using bolts (G2). Tighten bolts securely.


To Adjust:

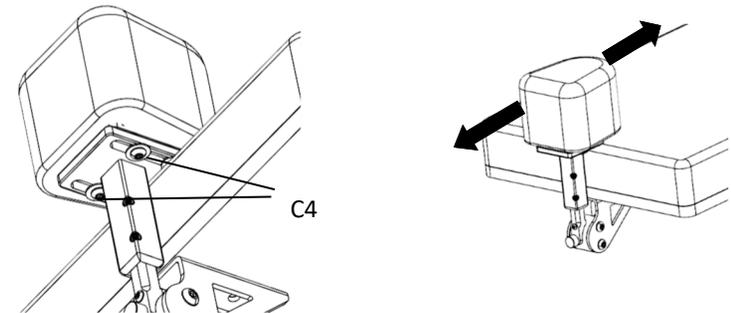Depth: Loosen bolts (C4). Slide to desired position and retighten bolts securely.

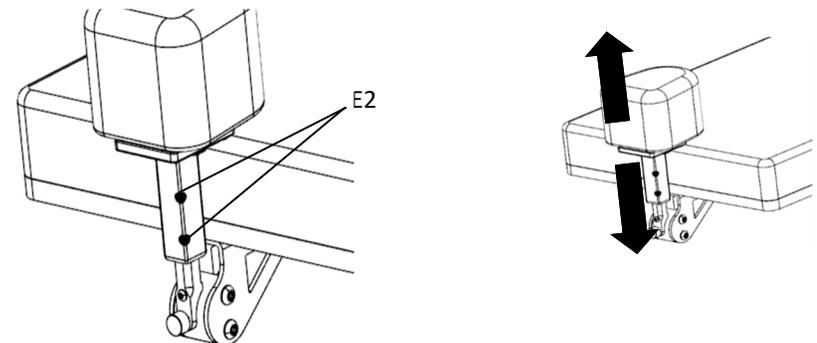
Height: Loosen bolts (E2). Slide to desired position and retighten bolts securely.

TO OPERATE:

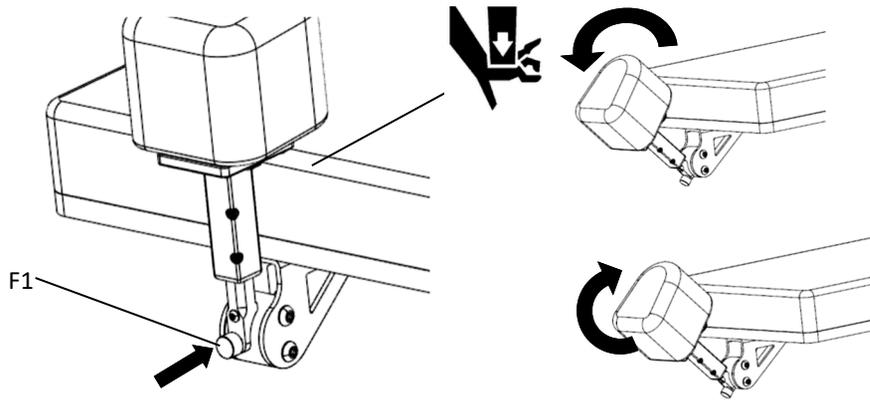Depress button (F1) to unlock device.

Care and Cleaning

 Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth.

2. Check all fasteners and retighten as needed.

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

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