


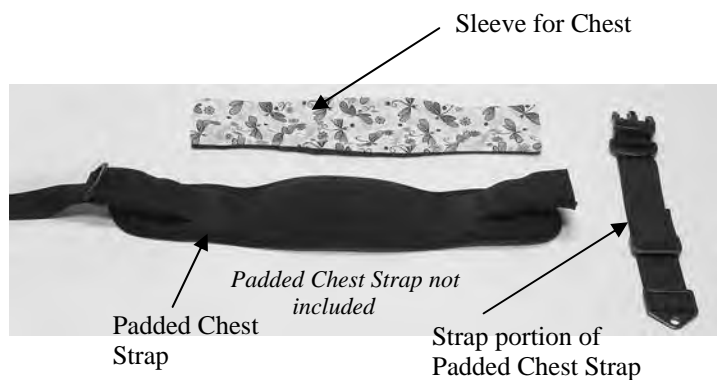


Chest Strap TheraSleeves

-  Caution: Read Instructions before operating device.
-  Caution: Device should be installed by a trained wheelchair technician.
-  Caution: Device should be used for positioning only.


Product Identification



To Install:




Step 1:

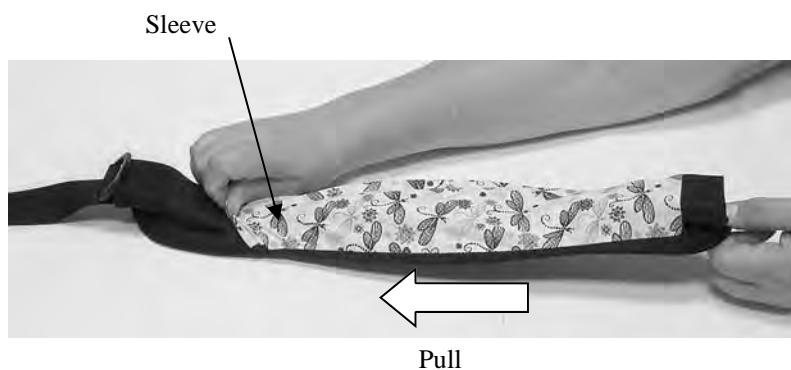

Care and Cleaning

1. Machine wash using warm water and a mild detergent. Tumble dry low. Do not use chlorine bleaches or fabric softeners.

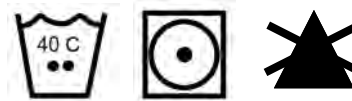

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

Document 08674
Revision A 12-2015



 Therafin Corporation
9450 W Laraway Road
Frankfort, IL 60423 USA
800-843-7234
www.theraфин.com



EC	REP
----	-----

EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands


HIP BELTS & CHEST STRAPS

*Parts included vary depending on which belt you have.
You may not have all parts shown here.*


METAL ATTACHING CLIPS

Depending on width of strap, thread strap as shown.

Metal Attaching Clip & 1" Or 1.5" Strap

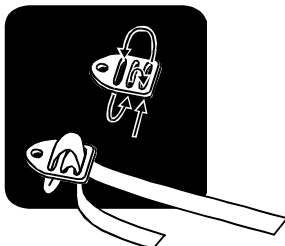


Metal Attaching Clip & 2" Strap

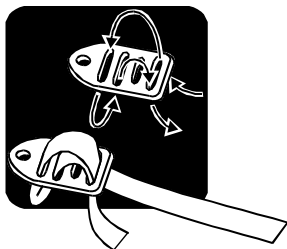


CAM LOCK BUCKLES

Thread strap through buckle and push top down to hold strap in place.

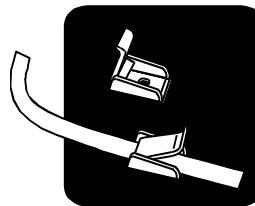


3-BAR SLIDING CLIPS

Use 3-Bar Sliding Clips to loop straps around chair frame.

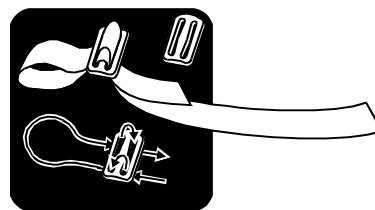


Use 3-Bar Sliding Clips with Metal Attaching Clips.

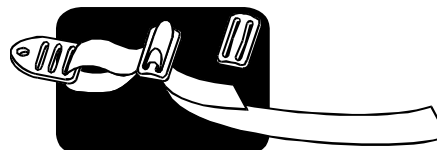

HINGED COLLARS

Attach Hinged Collars to chair frame. Attach Metal Attaching Clips or Cam Lock Buckles to Hinged Collars.

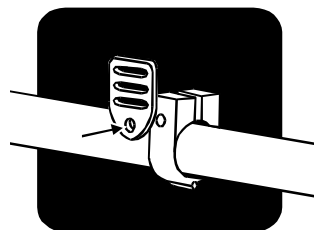

9450 W Laraway Rd • Frankfort, IL 60423
Customer 800-843-7234 • Fax 888-479-1515
www.TheraFin.com • info@TheraFin.com