


FlexSure Feet Instructions

Small Pair	31765
Medium Pair	31766
Large Pair	31767
X-Large Pair	31768

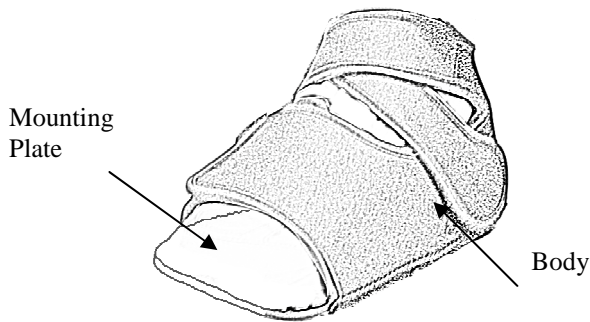

Install

Step 1 : Determine Location of the device on the users wheelchair footplate

Step 2 : Remove Mounting Plate and mark hole locations on the footplate. Drill all holes with 1/4" drill bit.




Step 3 : Place Mounting Plate back in Body and secure with included bolts and nuts.

08627

Operation

After inserting users foot, close front flaps. Then cross the upper straps and secure with hook and loop closure.

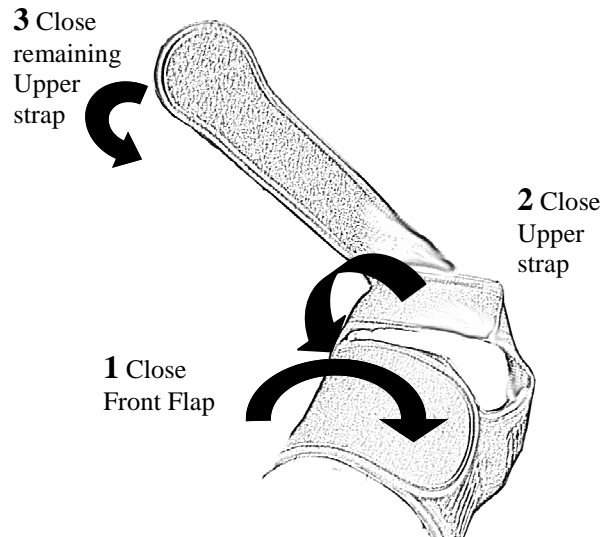

Over time, the hook surface can pick up lint and other debris that can affect its ability to grip the loop surface. Cleaning debris out of the hooks is similar to cleaning hairbrushes – and using a stiff brush or another piece of hook are the best methods. It can take some effort because the hooks in the tape are designed to grab. The optimal solution is prevention: the hook portion should be covered at all times, so it does not grab onto anything it should not – like a blanket or hair. Always engage the hook and loop before washing.



9450 W. Laraway Road • Frankfort, IL 60423
800 -843 -7234 • Fax: 888-479-1515
www.Therafin.com • info@Therafin.com

FlexSure Feet Padded Insert Instructions

Small Padded Insert Pair	31761
Medium Padded Insert Pair	31762
Large Padded Insert Pair	31763
X-Large Padded Insert Pair	31764



Install

Step 1 : Remove adhesive hook from Pad. Peel backing and place adhesive side on Mounting Plate

Step 2 : Install Pad on hook from Step 1

Over time, the hook surface can pick up lint and other debris that can affect its ability to grip the loop surface. Cleaning debris out of the hooks is similar to cleaning hairbrushes – and using a stiff brush or another piece of hook are the best methods. It can take some effort because the hooks in the tape are designed to grab. The optimal solution is prevention: the hook portion should be covered at all times, so it does not grab onto anything it should not – like a blanket or hair. Always engage the hook and loop before washing.

08627



9450 W. Laraway Road • Frankfort, IL 60423
800 -843 -7234 • Fax: 888-479-1515
www.TheraFin.com • info@TheraFin.com