



BASIC HEADREST HARDWARE

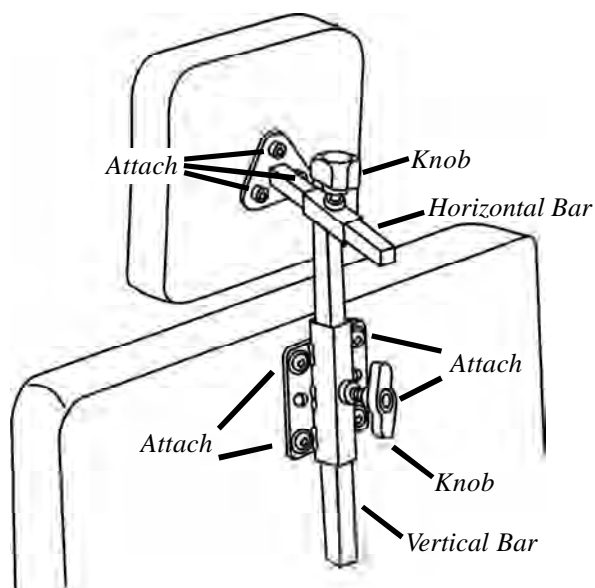

Attach

Step 1: Attach the Mounting Bracket to the chair with the 4 button head screws and washers provided.

Step 2: Attach your headrest pad (not included) to the Horizontal Bar with the 3 cap screws provided. The triangular end must be positioned correctly (see below).

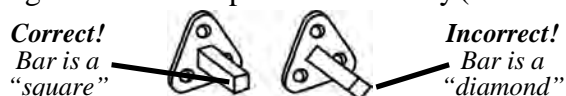


Adjust

Step 3: Slide the long t-shaped Vertical Bar into the Mounting Bracket. Tighten the Knob to keep it at the desired vertical position.

Step 4: Slide the shorter Horizontal Bar and headrest pad into the Vertical Bar. Tighten the Knob to keep it at the desired horizontal position.

08603

VERTICAL HEADREST HARDWARE

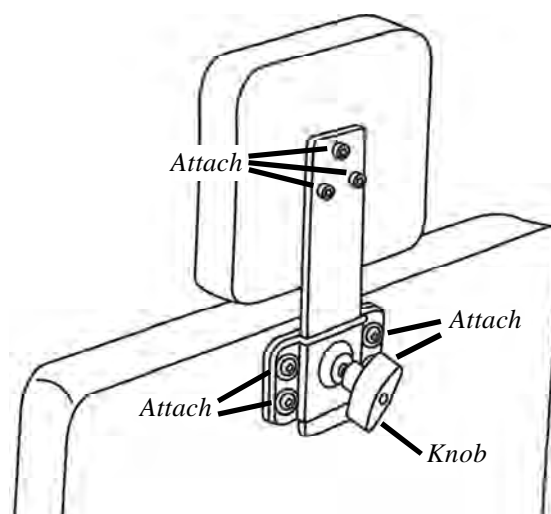

Attach

Step 1: Attach the Mounting Bracket and matching Spacer to the chair with the 4 button head screws and washers provided.

Step 2: Attach your headrest pad (not included) to the Stainless Steel Upright with the 3 cap screws provided.

Adjust

Step 3: Adjust the headrest pad to the desired vertical position and and tighten the Knob.

08604



9450 W Laraway Rd
Frankfort, IL 60423
www.Therafin.com

800-843-7234
Fax 888-479-1515
info@Therafin.com