


# Swingaway Lateral Trunk Support Kit And Lateral Pad\*

\*Lateral Pad is optional

## PARTS DESCRIPTION

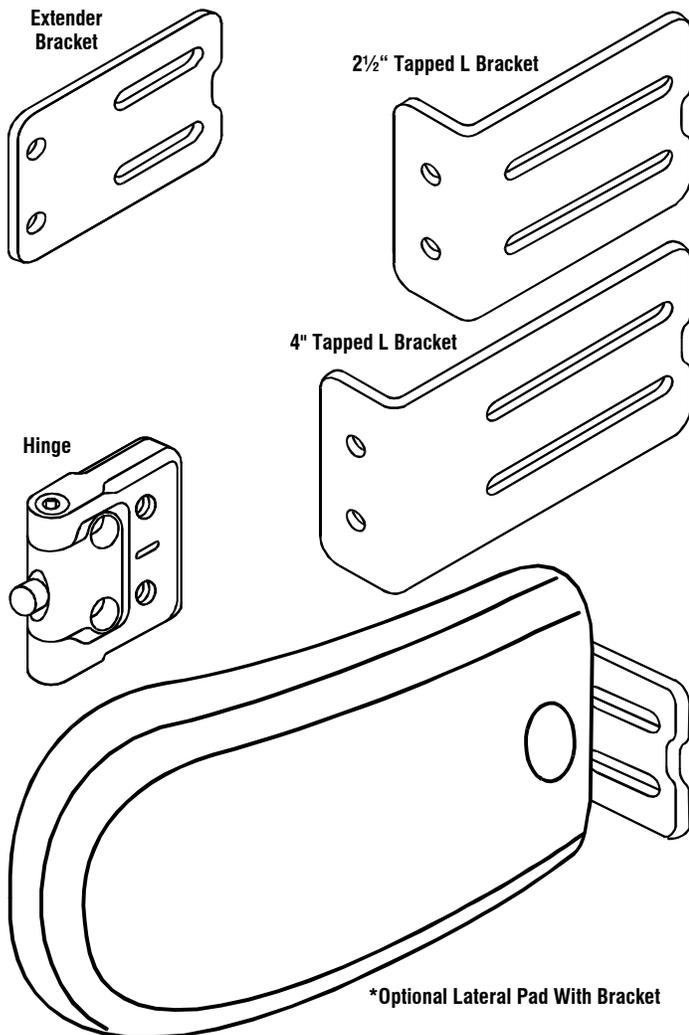


## COMPONENT ASSEMBLY


This information is general, as the flexibility of this kit allows you to configure components in many different ways. The brackets can be used in any number of different positions, and the slots in the brackets allow for extra adjustment. Use these example illustrations to determine the best way to set up the kit based on the user's particular needs and equipment.

## SWINGAWAY ASSEMBLY

1. Attach Hinge (A) to Back Bracket (B) with two Socket Head Cap Screws (C).

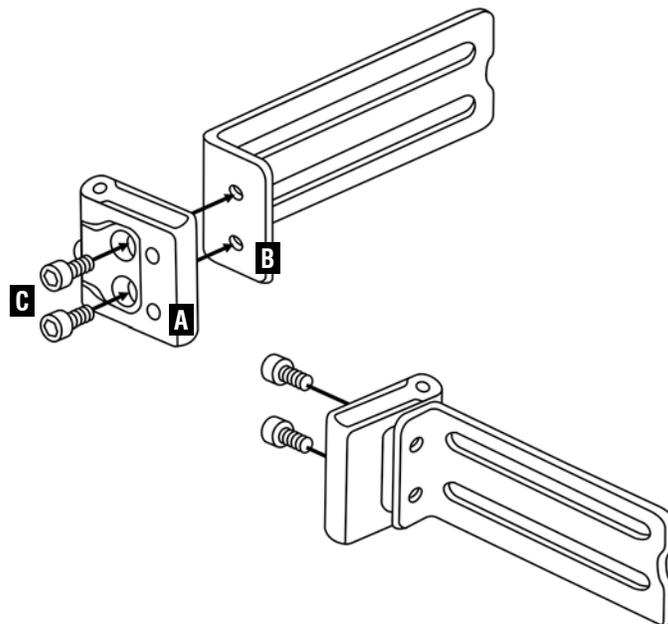


2. Attach Hinge Bracket (E) to Hinge (A) with two Button Head Cap Screws (F) and two Flat Washers (G).

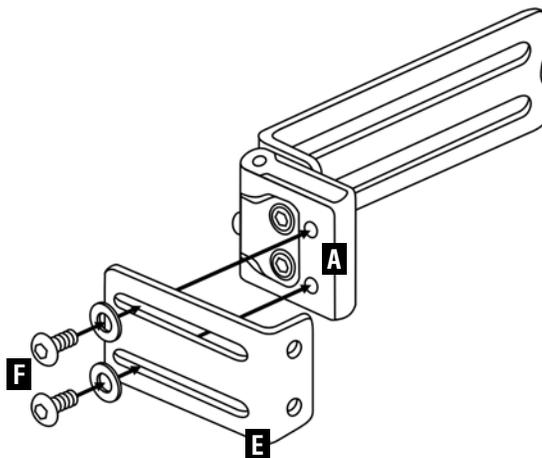


## USING SWINGAWAY

The **Red Push Button** allows the Lateral Pad to flip out of the way when getting into or out of the wheelchair.

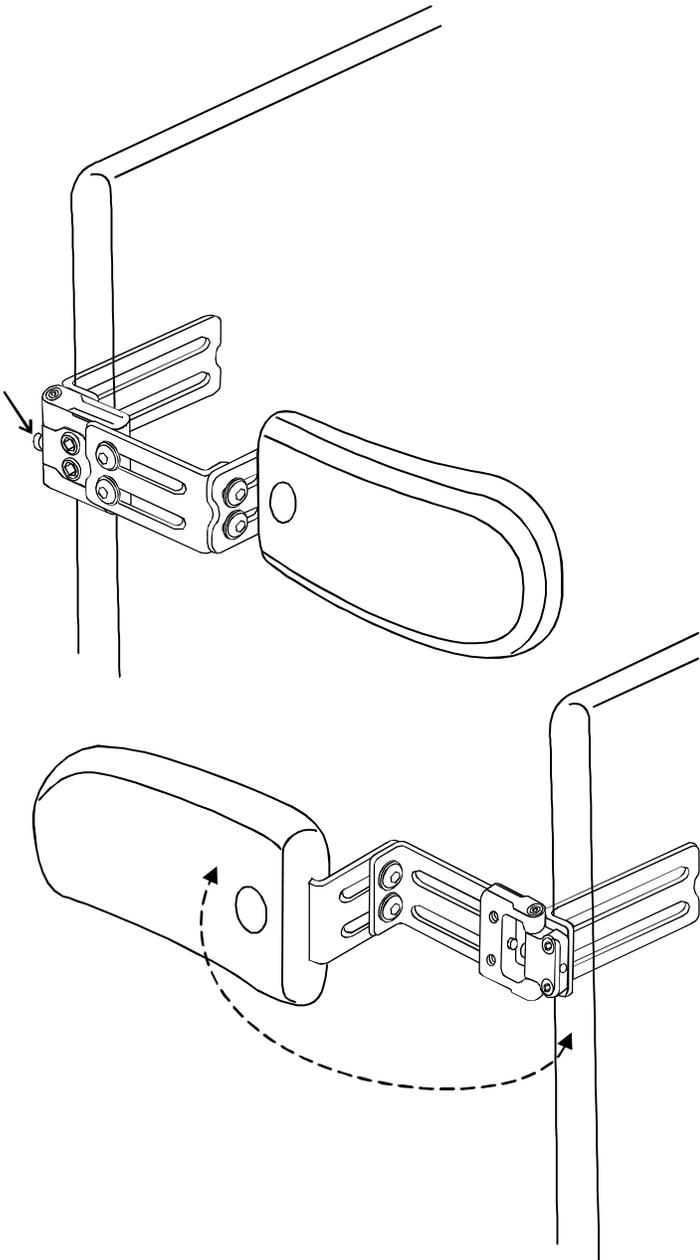


## FIXED VERSION

The kit may be used without the **Hinge**.

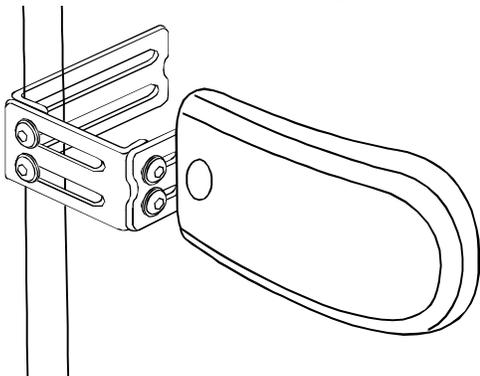


## REVERSIBLE LATERAL PAD (optional)

The pad is reversible on the bracket, for soft or firm support.

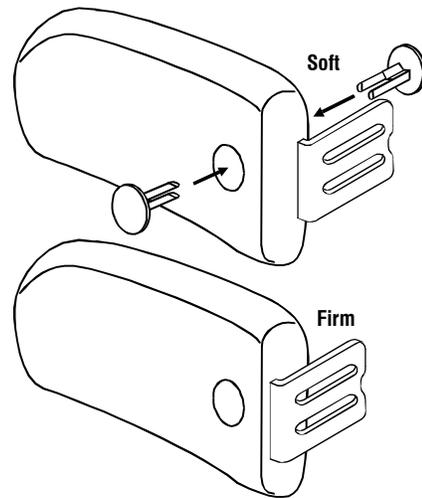


## PARTS AND ACCESSORIES

- ◆ Use Extender Bracket for extra adjustability in any direction
- ◆ Use Tarsus Multimount 1" Adaptor (order part# 32828) to connect the unit to a Tarsus multimount chair frame

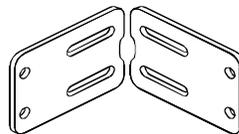

Extender Bracket can be used in any direction

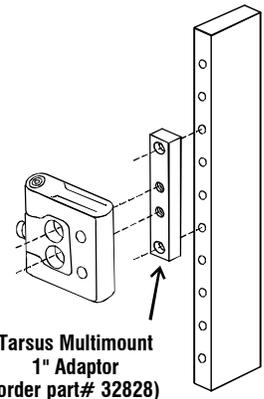

Tarsus Multimount 1" Adaptor (order part# 32828)

- ◆ Use Neoprene Sleeve (order part# 50135) to cover exposed bracket and hardware
- ◆ See all our Lateral Hardware pieces in the [Positioning Replacement Parts](#) section of our Catalog