




J&L BACK HARDWARE

These instructions are general information, and should be used to determine the best way for you to set up the kit, based on your particular needs and equipment

Attach Hardware To Back Cushion

Step 1: Install the J&L Brackets on the back cushion. Adjust them so they hold the cushion in the desired position relative to the back canes. Tighten them onto the back cushion.

IMPORTANT: Squeeze the Rail Lock in place and install the bolt **STRAIGHT THROUGH IT!**

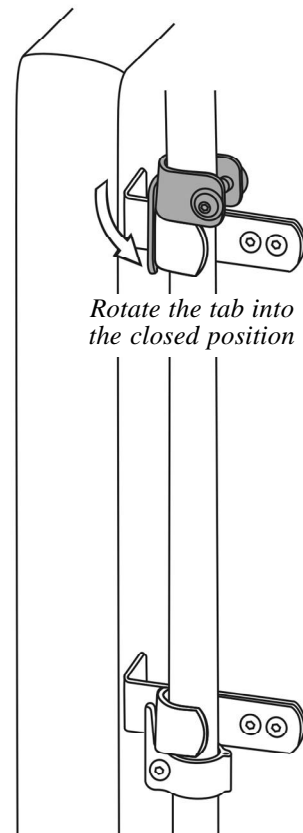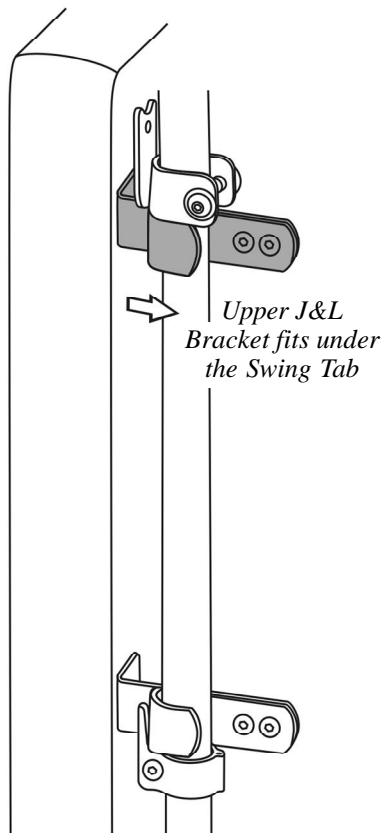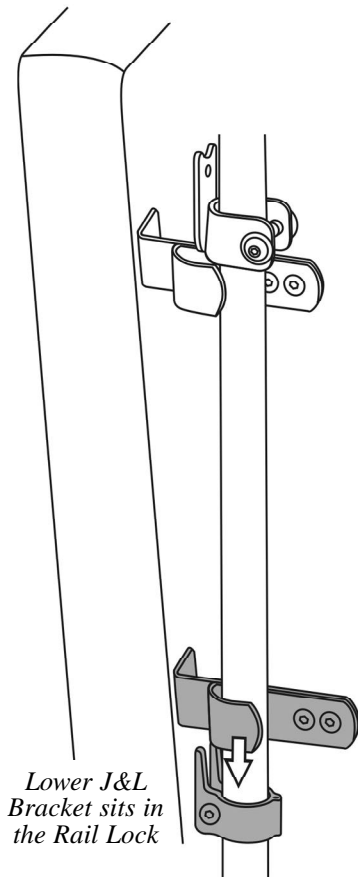

Attach Hardware To Chair

FIRST READ IMPORTANT NOTES BELOW!

Step 2: Install the Swing Tabs and Rail Locks on the back canes of the chair. With Rail Locks, use all 4 spacers for 7/8" tubes, or 2 for 1" tubes.

Position the Rail Locks to hold the lower J&L Brackets, and tighten them onto the canes. Then position the Swing Tabs directly above the upper J&L Brackets, and tighten them onto the canes.

Attach Back Cushion To Chair

Step 3: Tilt the cushion and set the lower J&L Brackets into the Rail Locks. Straighten the cushion so the upper J&L Brackets are positioned directly under the Swing Tabs. Rotate the tabs into the closed position to hold cushion in place.

IMPORTANT: DO NOT OVERTIGHTEN the Rail Locks! When attached to the frame, A GAP WILL REMAIN BETWEEN THE PIECES.

08610



9450 W Laraway Rd • Frankfort, IL 60423
Customer 800-843-7234 • Fax 888-479-1515
www.Therafin.com • info@Therafin.com