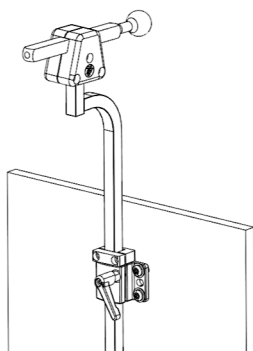
Multi-Axis Offset Hardware

Models:

50215

50215TOP

2. (en) Provide posterior head support to address fatigue or to prevent extension.
6. (da) Giv bageste hovedstøtte for at tackle træthed eller for at forhindre forlængelse.
10. (nl) Zorg voor achterzijde hoofdsteun om vermoeidheid tegen te gaan of om verlenging te voorkomen.
14. (fi) Tarjoa takaosan pään tuki väsymyksen korjaamiseksi tai venymisen estämiseksi.
18. (fr) Fournir un support de tête postérieur pour traiter la fatigue ou empêcher l'extension.
22. (de) Stellen Sie eine posteriore Kopfstütze bereit, um Ermüdung entgegenzuwirken oder eine Extension zu verhindern.
26. (it) Fornire un supporto per la testa posteriore per affrontare l'affaticamento o per prevenire l'estensione.
30. (no) Gi bakre hodestøtte for å adressere tretthet eller for å forhindre utvidelse.
34. (pt) Fornece suporte posterior para a cabeça para tratar a fadiga ou prevenir a extensão.
38. (es) Proporciono soporte posterior para la cabeza para abordar la fatiga o para evitar la extensión.
42. (sv) Ge bakre huvudstöd för att ta itu med trötthet eller för att förhindra förlängning.

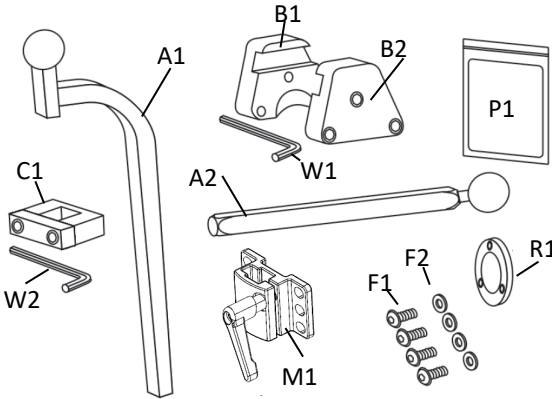
Multi-Axis Offset Headrest Hardware

English

-  Caution: Read Instructions before operating device.
-  Caution: Device should be installed by a trained wheelchair technician.
-  Caution: Device should be attached to a wheelchair back that conforms with ISO16840-3 or equivalent.
-  Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
-  Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

Product Identification

Components of the Multi-Axis Offset Headrest Device.


Clean before assembly and use


Components may have a protective coating to shield from environmental damage. Use wipe (P1) to clean components prior to assembly and use.

Installation and Assembly

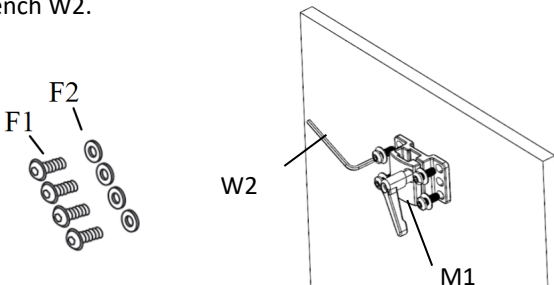
1. Attach M1 to Wheelchair back using F1 and F2. Tighten securely with wrench W2.

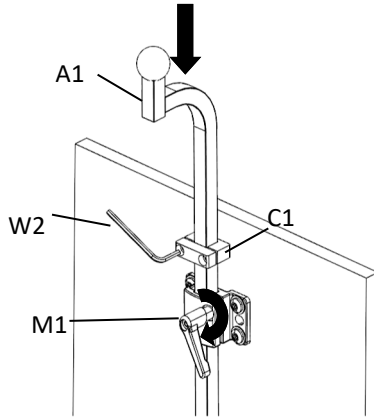
2. Place C1 onto A1 and insert A1 into M1. Tighten Knob.

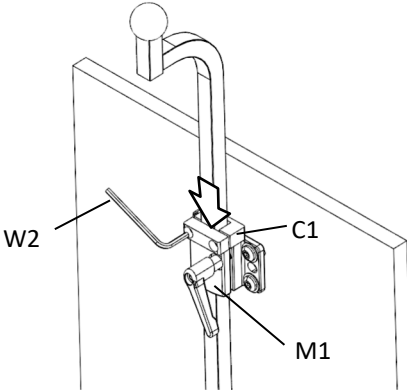3. Place C1 in desired position and tighten bolts with wrench W2.

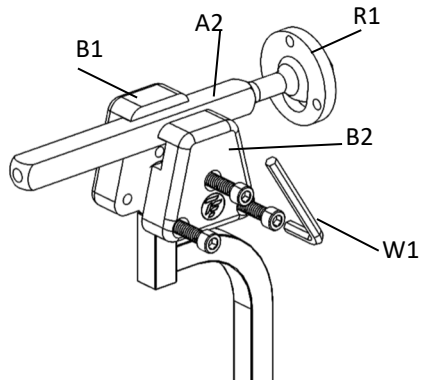
4. Assemble B1, B2, R1 and A2 as shown. Tighten Bolts with Wrench W1.

Multi-Axis Offset Headrest Hardware

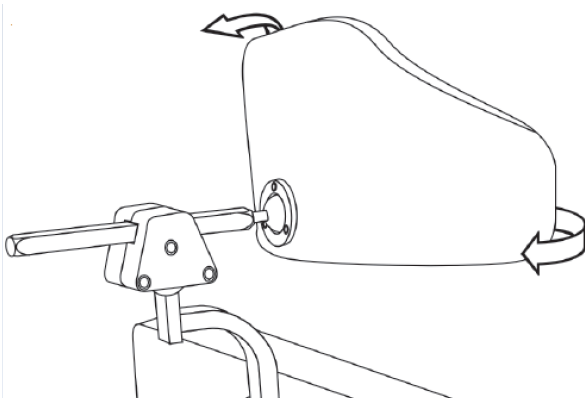5. Attach headrest. (See Headrest Instructions)

Adjustment

Caution: Device should be installed by a trained wheelchair technician.

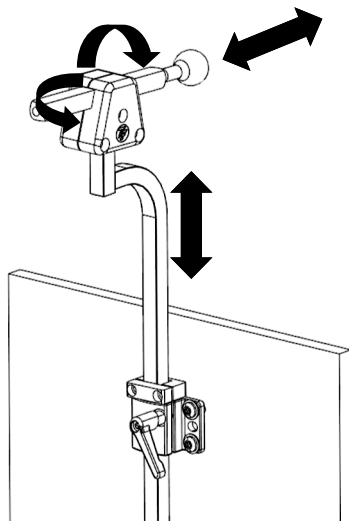
Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.

Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

Adjustments shown below. Re-tighten all fasteners after desired position is obtained.

Care and Cleaning

 Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth. Air Dry.

2. Check all fasteners and retighten as needed.

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

Notices

If patient experiences any skin irritation or severe discomfort contact your healthcare professional or medical technician.

Document 08748

Revision G 7-2022

If any serious incident or injury occurs please contact Therafin Corp and the competent authority for your country.

Product is made of materials which can be disposed of safely without special precautions.

Therafin Corporation
9450 W Laraway Road
Frankfort, IL 60423 USA
800-843-7234
www.therafin.com

MedEnvoy Global B.V.

Prinses Margrietplantsoen 33- Suite 123
2595 AM The Hague
The Netherlands

Multi-Axis Offset Headrest Hardware

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Kontakta Therafin Corp och den behöriga myndigheten i ditt land om någon allvarlig incident eller skada inträffar.