
Products

30785

30786

30787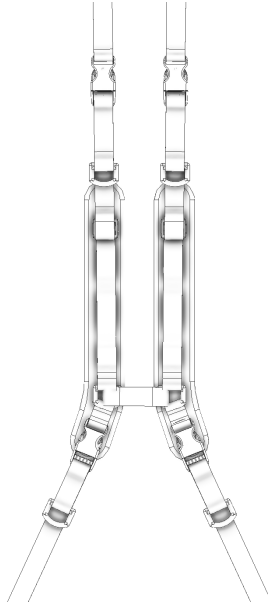

Document 18750
Revision A 4-2023

2. English

To provide support to the torso of a wheelchair user to promote upright posture. Not intended as a restraint in a moving vehicle.

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

X Harness

English

Image below shows installed device.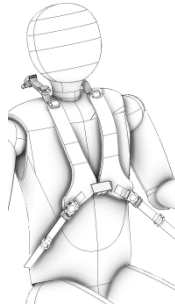


Product Identification

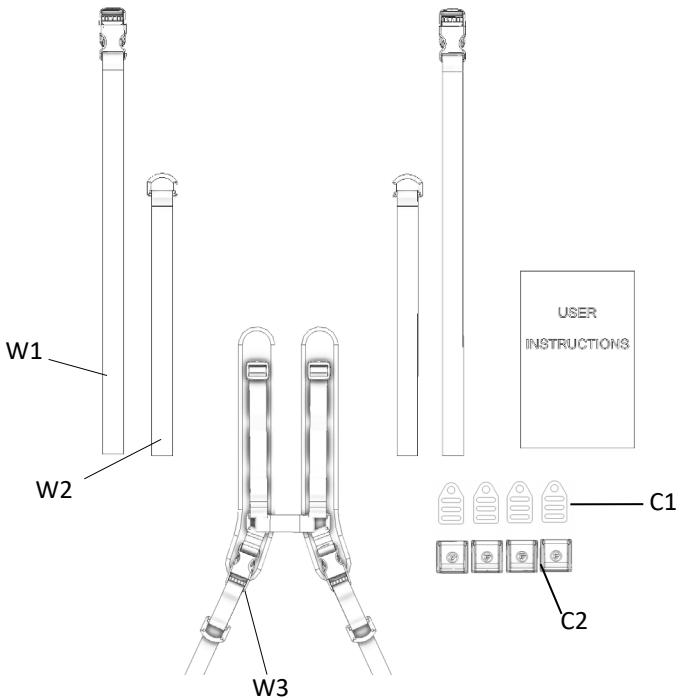X Harness

Setup:

Choose Front Pull or Rear Pull

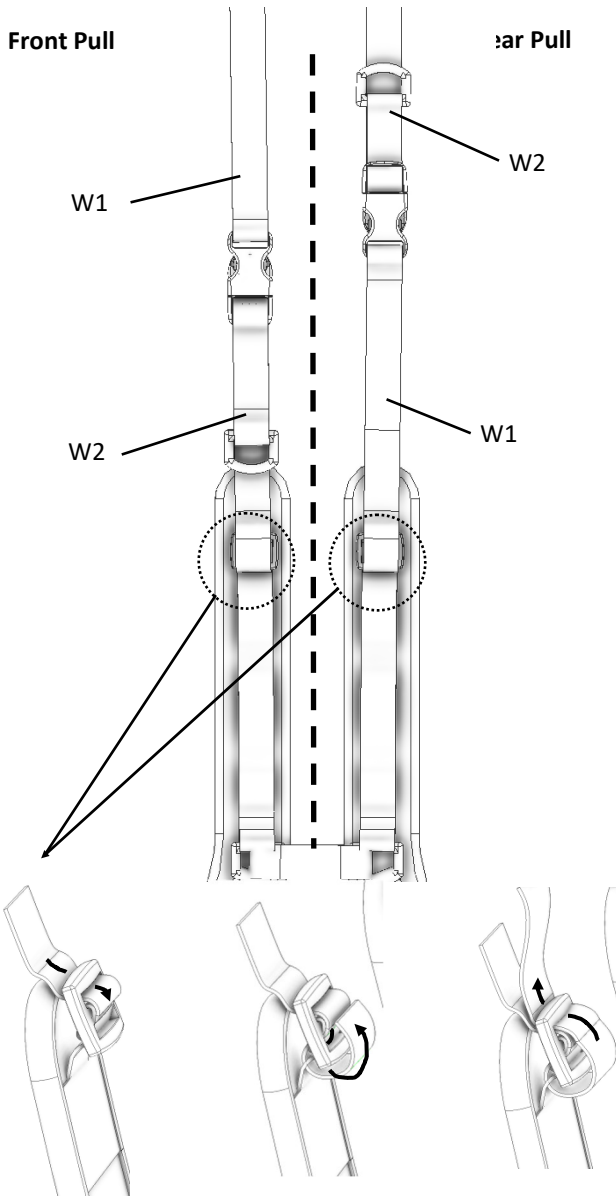X Harness

Caution: Read Instructions before operating device.

Caution: Device should be installed by a trained wheelchair technician.

Caution: Device should be attached to a wheelchair back that conforms with ISO16840-3 or equivalent.

Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.

Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

Warning: This device is NOT intended as an occupant restraint in a moving vehicle. This device is for positioning use only and should not be relied upon as a transit safety restraint. Use only WC-3 tested products for occupant restraint in a moving vehicle.



Warning: This device should be worn properly. Position the device properly to avoid choking hazard. Device should always be used with a hip positioning belt.

To Install:

Step 1: Identify attaching method:

2 Clips and 2 Cams (C1 and C2)

Step 2: Feed webbing (W1) through clips (C1) and W3 through cams (C2).

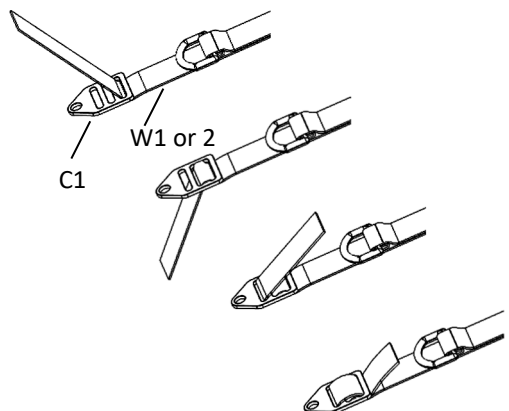
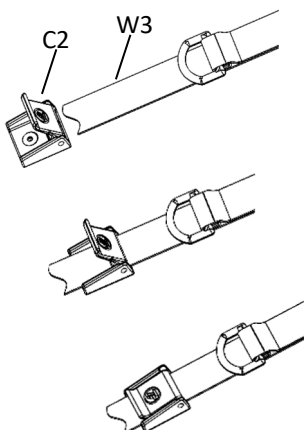
X Harness



Step 3: Position device on patient. Do not place too high near neck.

Warning: This device should be worn properly. Position the device properly to avoid choking hazard. Device should always be used with a hip positioning belt.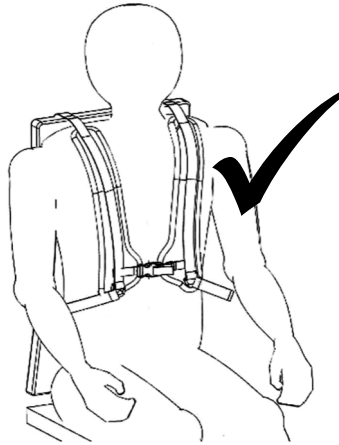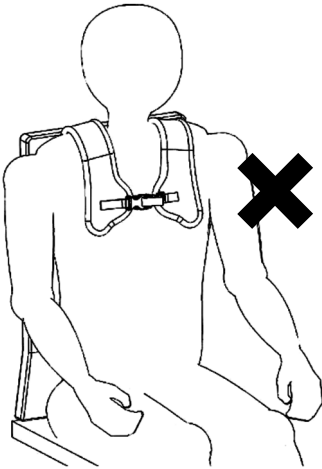

Step 4: Attach clips (C1) and cams (C2) to wheelchair back.
Tighten fasteners.

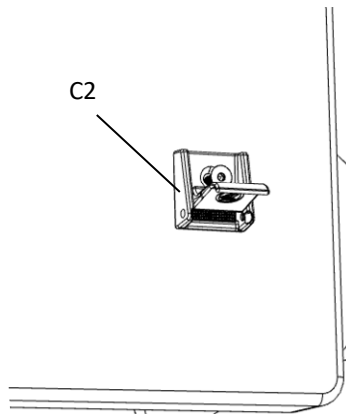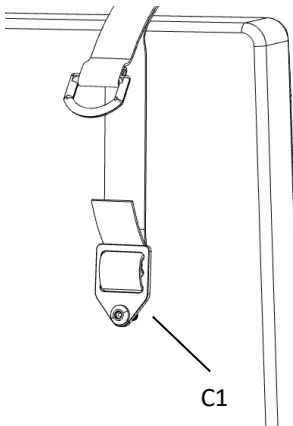
X Harness

To Adjust:

 **Warning:** This device should be worn properly. Position the device properly to avoid choking hazard. Device should always be used with a hip positioning belt.

Once all clips (C1) and cams (C2) are secured, tighten straps by pulling on D-Ring. Be sure Harness is NOT positioned too high near neck.

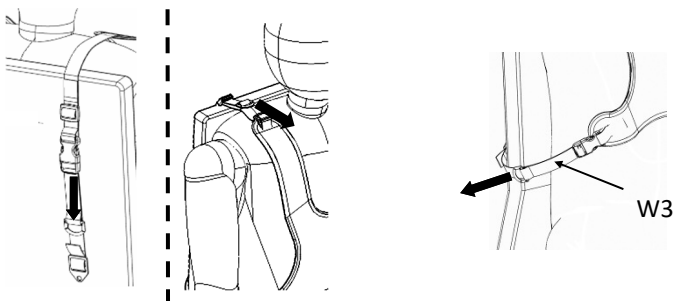Care and Cleaning

 **Caution:** Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth. Air Dry.

Document 18750
Revision A 4-2023

2. Check all fasteners and retighten as needed.

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

Therafin Corporation
9450 W Laraway Road
Frankfort, IL 60423 USA
800-843-7234
www.therafin.com

Notices

If patient experiences any skin irritation or severe discomfort contact your healthcare professional or medical technician.

If any serious incident or injury occurs please contact Therafin Corp and the competent authority for your country

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Si ocurre algún incidente o lesión grave, comuníquese con Therafin Corp y la autoridad competente de su país.