




2. English

Unpadded Chest Strap

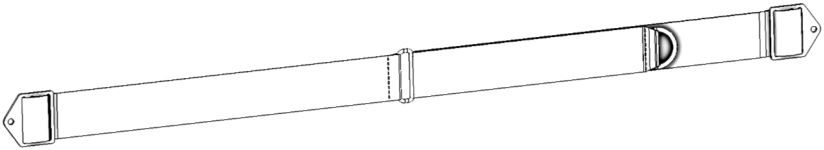


Document 18500

Revision A 06-2023

Unpadded Chest Strap


English

Image below shows installed device.

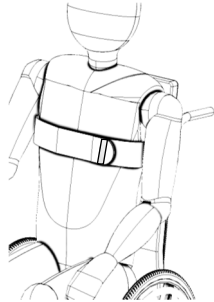

Use: To provide Torso support to a wheelchair user to address fatigue or lack of muscular control. Assists with upright posture by supporting the upper body.

Product Identification

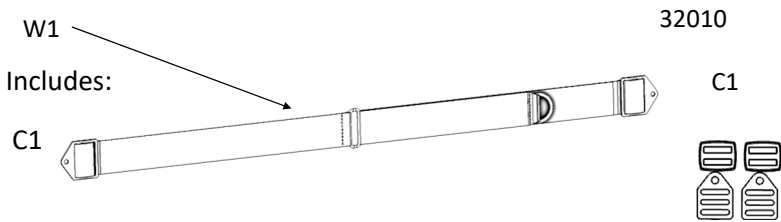

Caution: Read Instructions before operating device.



Caution: Device should be installed by a trained wheelchair technician.



Caution: Device should be attached to a wheelchair back that conforms with ISO16840-3 or equivalent.



Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.



Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.



Warning: This device is NOT intended as an occupant restraint in a moving vehicle. This device is for positioning use only and should not be relied upon as a transit safety restraint. Use only WC-3 tested products for occupant restraint in a moving vehicle.



Warning: This device should be worn properly. Position the device properly to avoid choking hazard. Device should always be used with a hip positioning belt.


Unpadded Chest Strap

To Install:

Step 1: Identify attaching method:

2 Clips (C1) OR 2 Clips and 2 Cams (C2)


Step 2: Feed webbing (W1) through clips (C1) as shown.

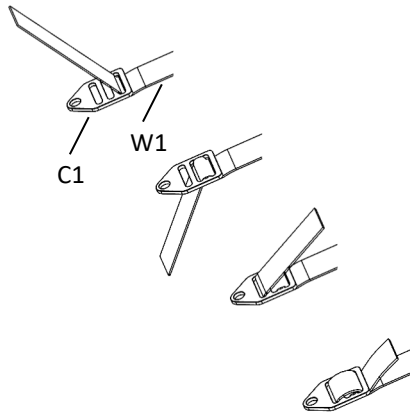

Step 3: Position device on patient. Do not place too high near neck.





Warning: This device should be worn properly. Position the device properly to avoid choking hazard. Device should always be used with a hip positioning belt.

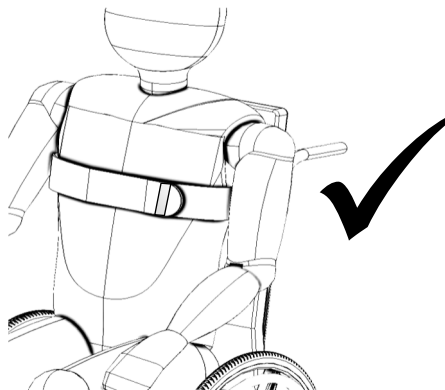


Unpadded Chest Strap

Step 4: Attach clips (C1) and cams (C2) to wheelchair back.
Tighten fasteners.

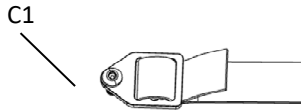

 **Warning:** This device should be worn properly. Position the device properly to avoid choking hazard. Device should always be used with a hip positioning belt.

Care and Cleaning

 **Caution:** Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth.



2. Check all fasteners and retighten as needed.

Notices

If patient experiences any skin irritation or severe discomfort contact your healthcare professional or medical technician.

If any serious incident or injury occurs please contact Therafin Corp and the competent authority for your country

Product is made of materials which can be disposed of safely without special precautions.

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

Document 18500
Revision A 06-2023



Therafin Corporation
9450 W Laraway Road
Frankfort, IL 60423 USA
800-843-7234
www.therafin.com