2. English

6. Dansk
Danish

10. Nederlands
Dutch

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18. Français
French

22. Deutsch
German

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30. Norsk
Norwegian

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38. Español
Spanish

42. Svensk
Swedish

Document 07100
Revision A 06-2021

 Therafin Corporation
9450 W Laraway Road
Frankfort, IL 60423 USA
800-843-7234
www.therafin.com

EC	REP
----	-----

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands


Document 07100
Revision A 06-2021

TheraFit Chest Strap

English

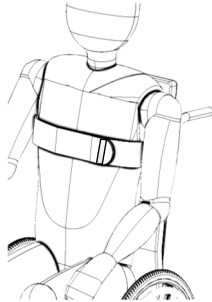Image below shows installed device.

Use: To provide Torso support to a wheelchair user to address fatigue or lack of muscular control. Assists with upright posture by supporting the upper body.

Product Identification

BLANK PAGE

W1
Includes:
C1 OR C2

60110, 60111, 60112

60113, 60114, 60115,
60123, 60124

60116, 60117, 60118,
60147, 60148, 60149

60119, 60120, 60121,
60150, 60151, 60152

60153, 60154, 60155

60157, 60158, 60159

C1

C2

The diagram shows six different models of the TheraFit Chest Strap, each with a unique shape and strap configuration. A bracket on the left groups them under 'W1 Includes: C1 OR C2'. Below the models, there are two columns of icons labeled 'C1' and 'C2'. The C1 column shows two icons of rectangular components. The C2 column shows a more complex set of components, including a central strap with buckles and two smaller rectangular components.

TheraFit Chest Strap

-  Caution: Read Instructions before operating device.
-  Caution: Device should be installed by a trained wheelchair technician.
-  Caution: Device should be attached to a wheelchair back that conforms with ISO16840-3 or equivalent.
-  Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
-  Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.
-  Warning: This device is NOT intended as an occupant restraint in a moving vehicle. This device is for positioning use only and should not be relied upon as a transit safety restraint. Use only WC-3 tested products for occupant restraint in a moving vehicle.
-  Warning: This device should be worn properly. Position the device properly to avoid choking hazard. Device should always be used with a hip positioning belt.

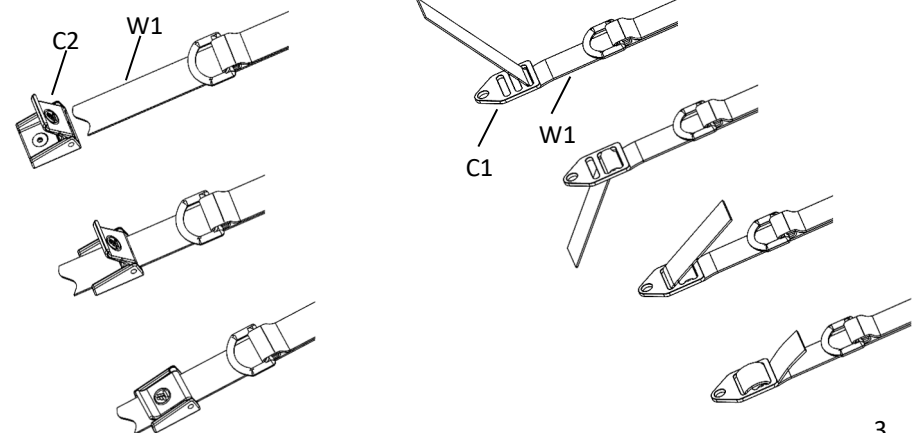
BLANK PAGE

To Install:

Step 1: Identify attaching method:

2 Clips (C1) OR 2 Clips and 2 Cams (C2)


Step 2: Feed webbing (W1) through clips (C1) and/or cams (C2) as shown.


TheraFit Chest Strap

Step 3: Position device on patient. Do not place too high near neck.

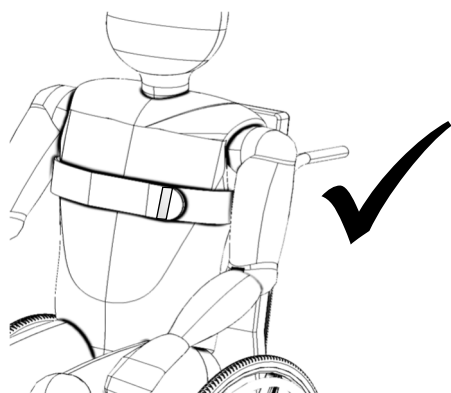 Warning: This device should be worn properly. Position the device properly to avoid choking hazard. Device should always be used with a hip positioning belt.

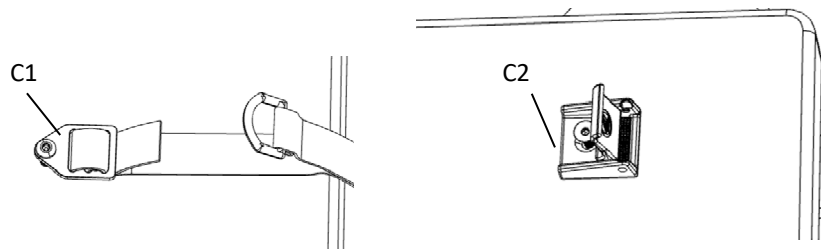Step 4: Attach clips (C1) and cams (C2) to wheelchair back. Tighten fasteners.

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Once all clips (C1) and cams (C2) are secured, tighten straps by pulling on D-Ring or tightening using clips. Be sure strap is NOT positioned too high near neck.

Care and Cleaning

 Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth.

2. Check all fasteners and retighten as needed.

Notices

If patient experiences any skin irritation or severe discomfort contact your healthcare professional or medical technician.

If any serious incident or injury occurs please contact Therafin Corp and the competent authority for your country

Product is made of materials which can be disposed of safely without special precautions.

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

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