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

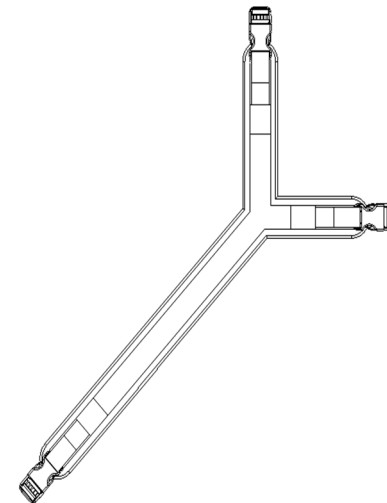
*Document 07958
Revision A 06-2021*

English

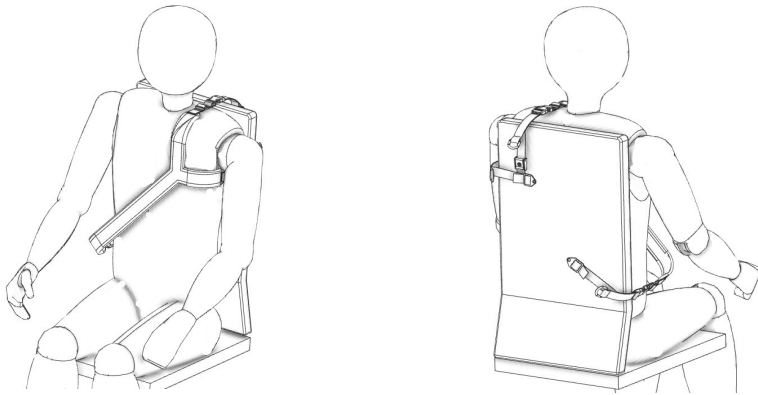Image below shows installed device.

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
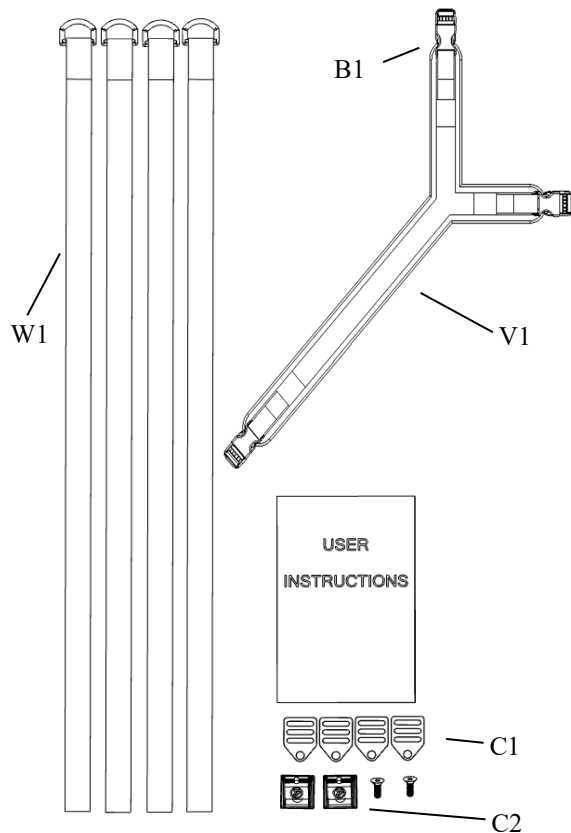
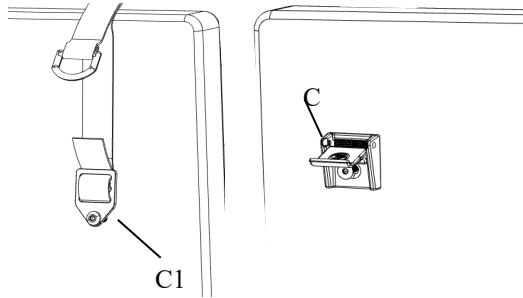
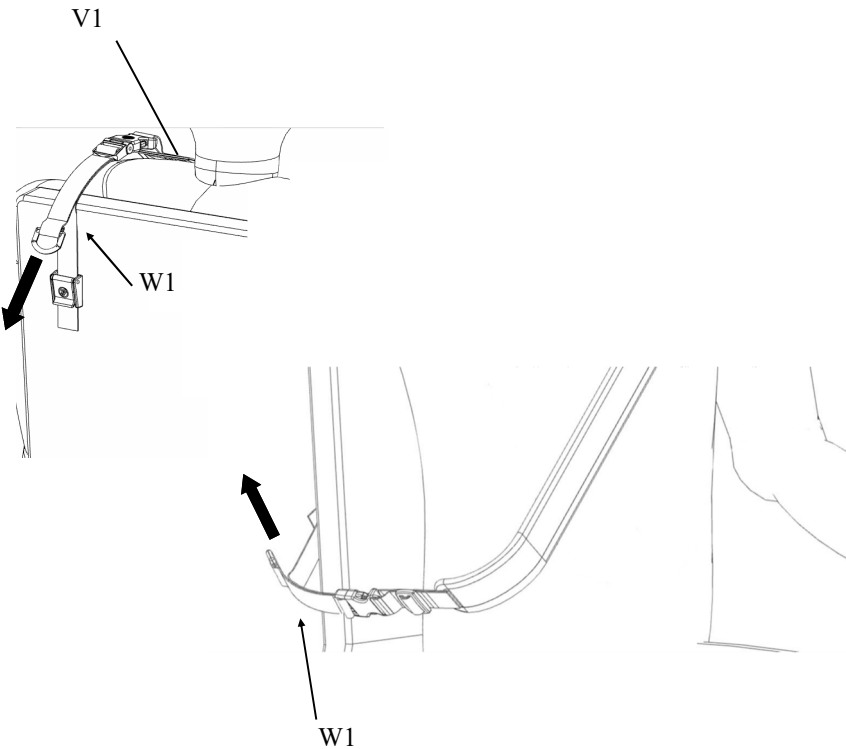
- ⚠** Caution: Device should be attached to a wheelchair back that conforms with ISO16840-3 or equivalent.
- ⚠** Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
- ⚠** Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.
- ⚠** Warning: This device should be worn properly. Position the device properly to avoid choking hazard. Device should always be used with a hip positioning belt.
- ⚠** Warning: This device is NOT intended as an occupant restraint in a moving vehicle. This device is for positioning use only and should not be relied upon as a transit safety restraint. Use only WC-3 tested products for occupant restraint in a moving vehicle.

To Install:

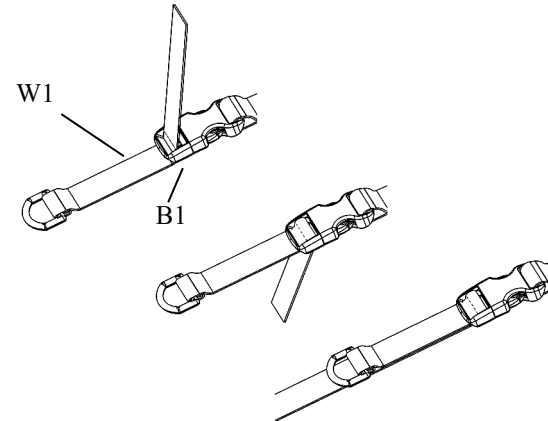
- Step 1:** Select attaching method:
2 of Clips (C1) and 2 of Cams (C2) , or 4 Clips (C1)
- Step 2:** Feed webbing (W1) through buckles (B1) as shown.

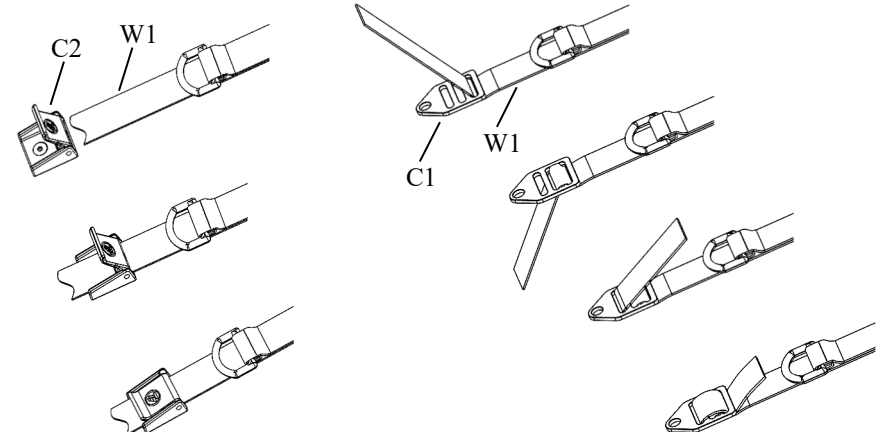- Step 3:** Feed webbing (W1) through clips (C1) and cams (C2) as shown.

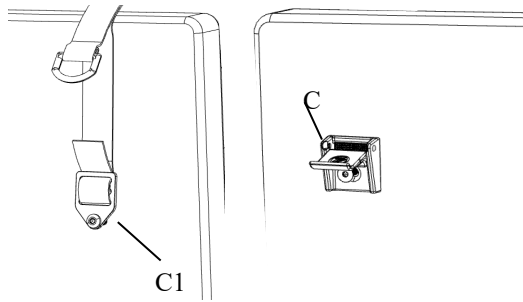
Step 4: Attach clips (C1) and cams (C2) to wheelchair back. Tighten fasteners. (Fasteners to secure C1 *NOT INCLUDED*)

To Adjust:

Warning: This device should be worn properly. Position the device properly to avoid choking hazard. Device should always be used with a hip positioning belt.

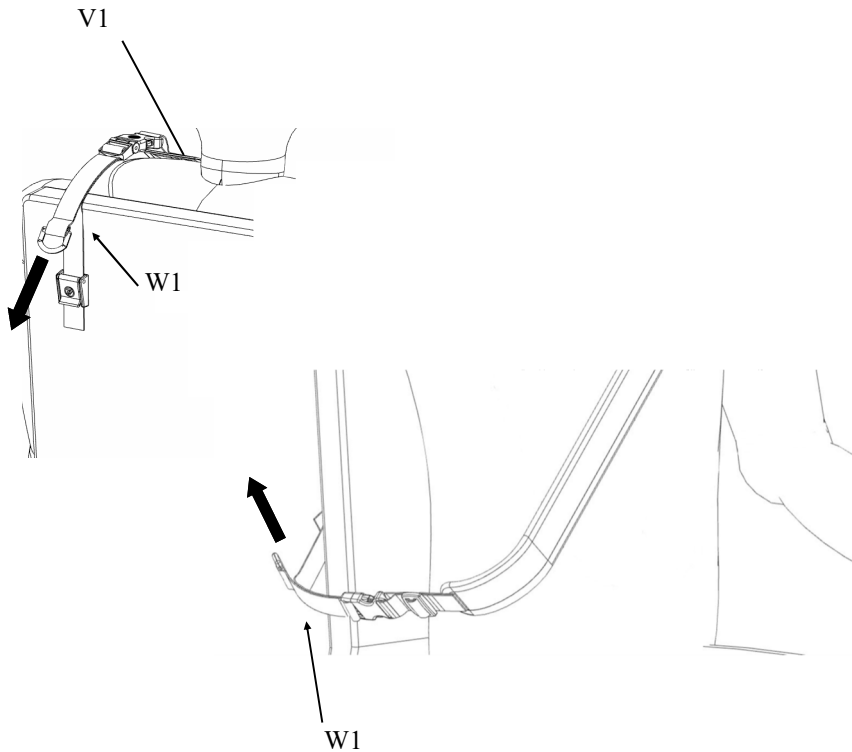Once all clips (C1) and cams (C2) are secured, tighten straps (W1) by pulling on D-Ring. Be sure vest is **NOT** positioned too high near neck.

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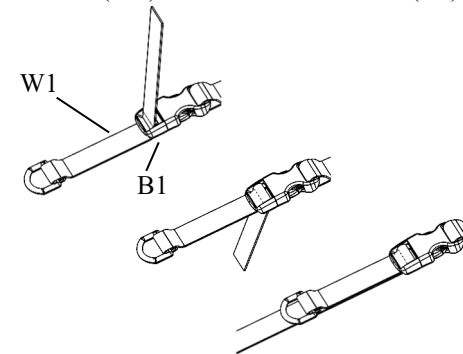
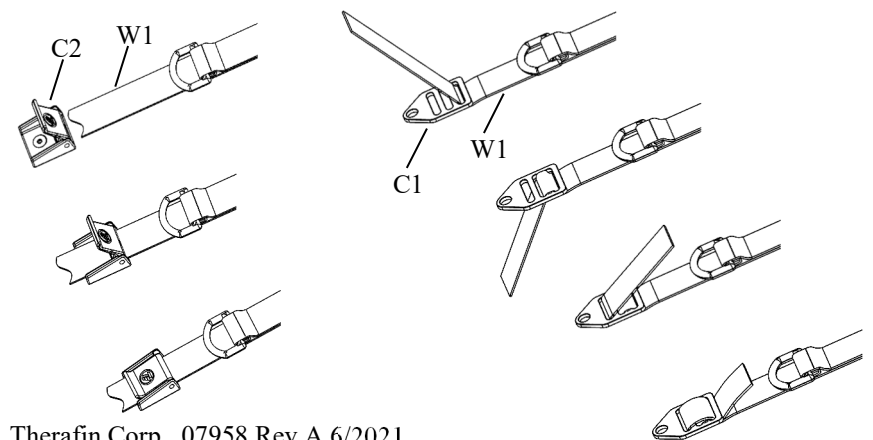
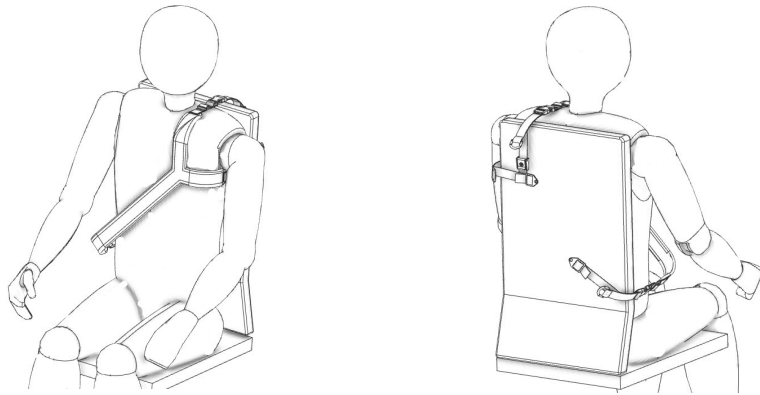
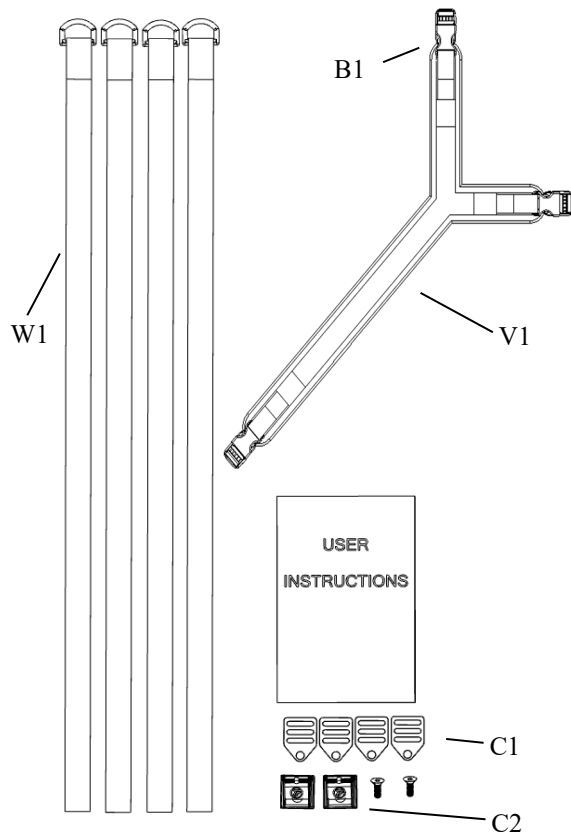
Care and Cleaning

Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth.
Air Dry.

2. Check all fasteners and retighten as needed.

Notices

If patient experiences any skin irritation or severe discomfort contact your healthcare professional or medical technician.

If any serious incident or injury occurs please contact Therafin Corp and the competent authority for your country

Product is made of materials which can be disposed of safely without special precautions.

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

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Therafin Corp. 07958 Rev A