WAVVERTENZE

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Owner's Instruction Manual

Please ensure that this booklet stays attached to the seating device. This booklet is the property of the consumer and contains important information and instructions.

Use: For proper foot positioning in a wheelchair

TABLE OF CONTENTS

General Information.....	2
Product Identification and Contents.....	3
Installation	4
Positioning Device Adjustment	6
Cleaning and Maintenance	7
Warranty	8

GENERAL INFORMATION

This Manual Part Number 17965 (07965) Revision C

Hours: Monday thru Thursday 7AM to 5:30PM (Central Time)
Phone toll-free 800-843-7234 Fax toll-free (24 hrs.) 888-479-1515
Email: info@therafin.com

Retain all instructions and manuals you that you received with your device.

POSITIONING DEVICE INFORMATION AND WARNINGS

WARNING

READ THE OWNERS MANUAL BEFORE USE.

WARNING

THIS PRODUCT IS FOR POSITIONING ONLY. IT SHOULD NOT BE RELIED ON FOR OCCUPANT RESTRAINT IN A MOVING VEHICLE

WARNING

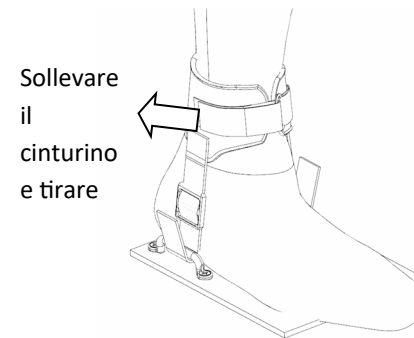
DO NOT CONTINUE TO USE THE DEVICE AFTER IDENTIFYING LOOSE OR BROKEN COMPONENTS. FAILURE TO COMPLY MAY RESULT IN INJURY TO THE USER.

WARNING

DO NOT MAKE CHANGES TO THE POSITIONING DEVICE UNLESS YOU HAVE CONSULTED YOUR AUTHORIZED SUPPLIER FIRST. FAILURE TO COMPLY MAY RESULT IN INJURY TO THE USER.

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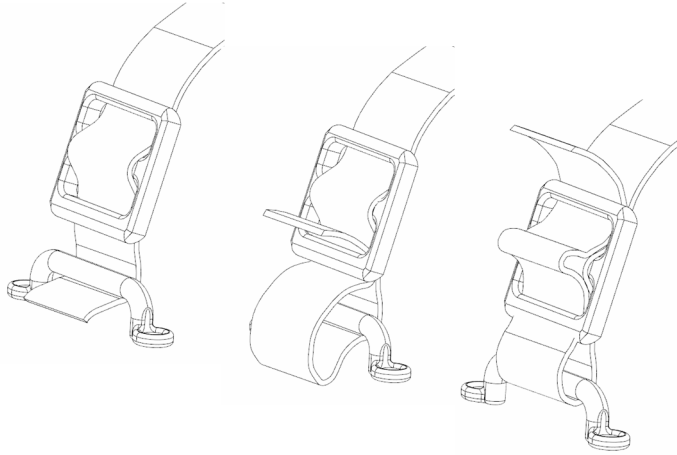
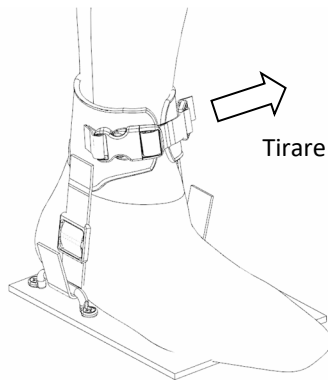
1. Fibbia:


PRODUCT IDENTIFICATION AND CONTENTS

The FootSure Ankle Support is available in two styles. They contain EITHER a Hook and loop closure or a Side Release Buckle closure. Identify which style of FootSure Ankle Supports you have with the chart and pictures below.

Side Release Buckle Closure

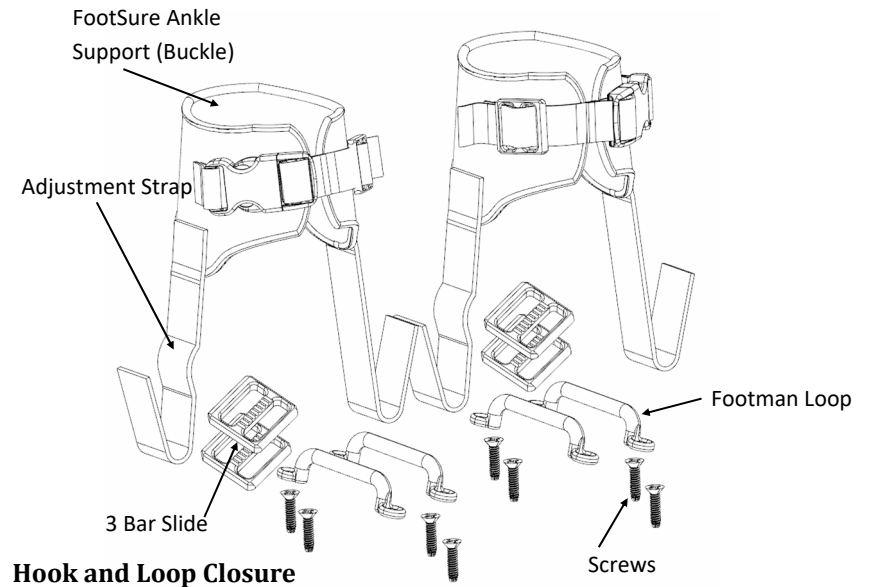CONTENTS OF SIDE RELEASE BUCKLE CLOSURE


Hook and Loop Closure

CONTENTS OF HOOK AND LOOP CLOSURE

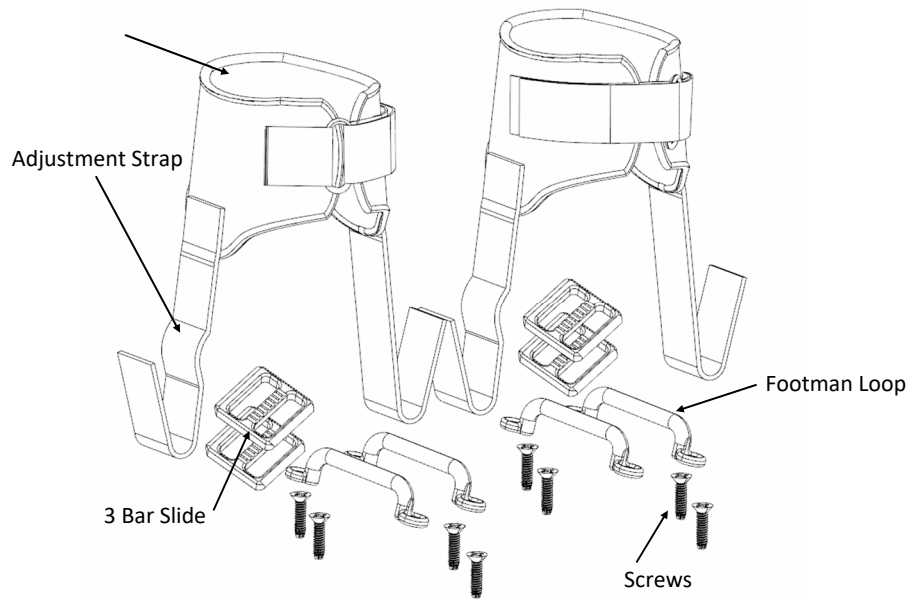
FootSure Ankle Support


INSTALLATION

STEP-1 POSITION AND ATTACH FOOTMAN LOOPS

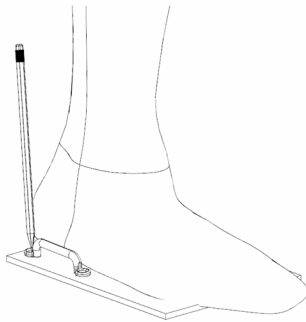
A. Position footman loops outside of the patient's shoes and mark the hole location.

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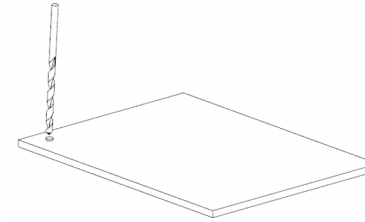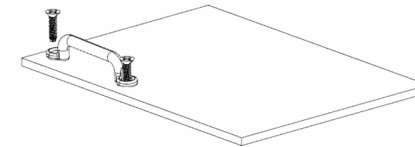
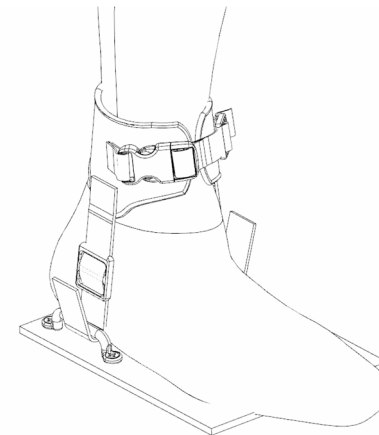
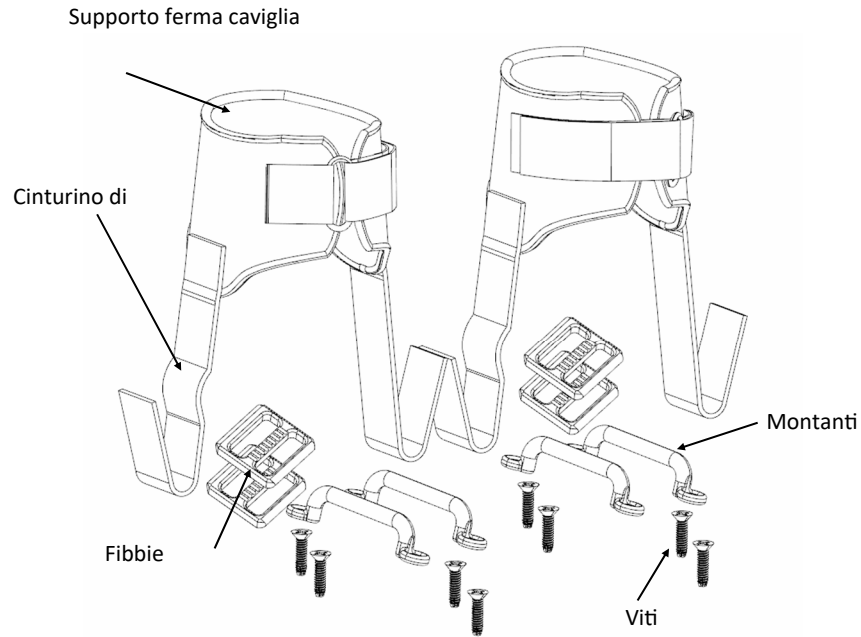
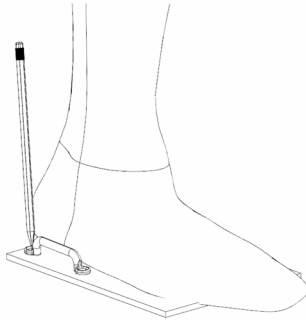
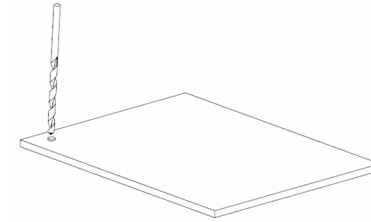
B. Drill holes using #17 or #16 drill bit. (nearest fractional size is 11/64")

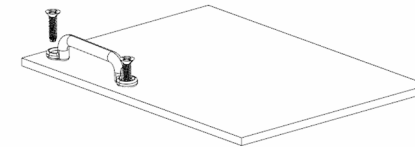
C. Fasten Footman Loops with supplied screws.


STEP-2 ATTACH FOOTSURE ANKLE SUPPORT STRAPS (HOOKS)

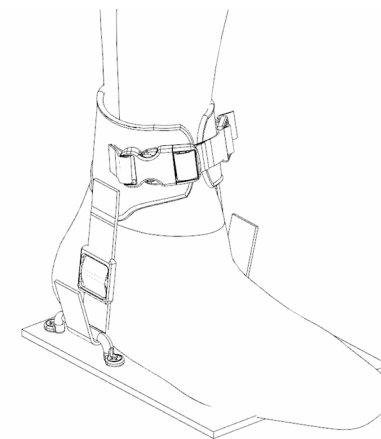A. Place FootSure Ankle Support on the patient's ankle and determine the length of adjustment strap. Place the 3 bar slide on the strap (as shown) and pass the strap under the footman loop.

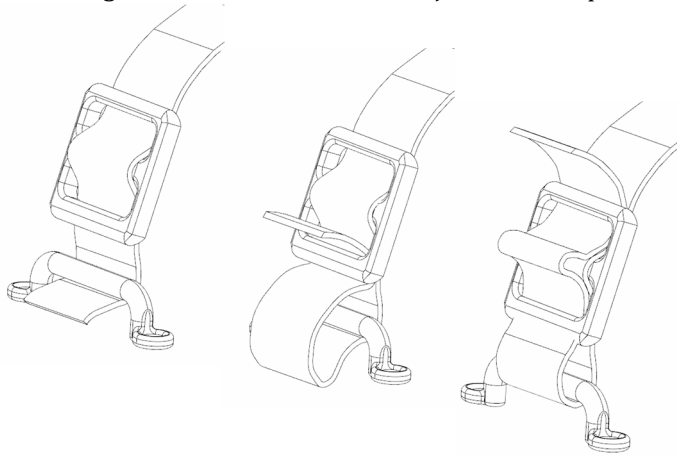B. Follow diagrams below to secure the adjustment strap

POSITIONING DEVICE ADJUSTMENT

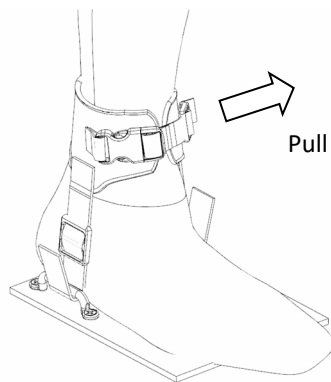
The FootSure Ankle Support has two main adjustments for growth and also to provide the correct user support.

Adjustments can be made through:

1. The adjustment straps as described in step 2 of the Installation Section. Adjustment of the closure buckles or straps

To make adjustments to buckle or strap closures:

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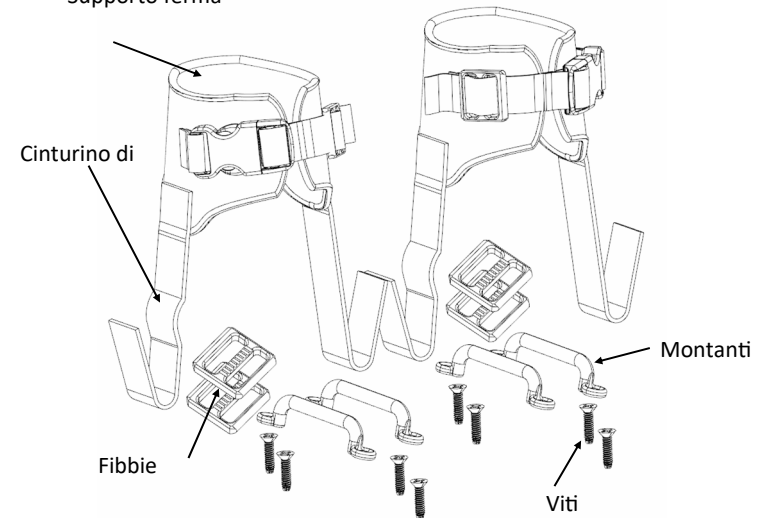
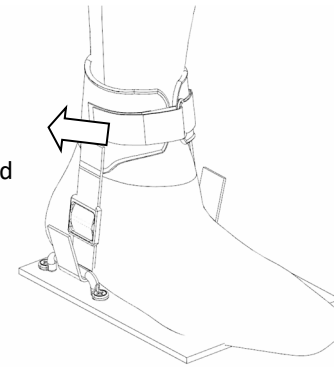
CLEANING:

UPHOLSTERY, WEBBING AND/OR FABRICS:

Surfaces may be wiped clean with mild soap and water. Do not use harsh detergents, cleaning agents or abrasives. Avoid the use of polishes that can make the surfaces slippery after cleaning.

POSITIONING COMPONENTS AND HARDWARE:

Wipe clean with mild soap and water. Do not use harsh detergents, cleaning agents or abrasives. Avoid the use of polishes that can make the surfaces slippery after cleaning. Do not submerge components underwater.

If you have any additional questions about how to clean or maintain your device please contact your Therafin dealer or agent.

MAINTENANCE:

WARNING

FAILURE TO FOLLOW THE MAINTENANCE BELOW MAY RESULT IN INIURY TO THE USER.

WARNING

DO NOT CONTINUE TO USE THE SYSTEM AFTER IDENTIFYING LOOSE OR BROKEN COMPONENTS OR FASTENERS:

Therafin recommends that all fasteners should be checked for wear every 3 months. Perform an inspection for loose bolts or broken components every 3 months. Loose fasteners should be retightened according to the installation instructions. Contact your Therafin or dealer supplier immediately to replace any broken or malfunctioning components.

WARNING

DO NOT CONTINUE TO USE THE SYSTEM AFTER IDENTIFYING LOOSE OR BROKEN COMPONENTS. REMOVABLE, ADJUSTABLE OR HINGED COMPONENTS.

Check the function of all moveable components during each use. Ensure they adjust and lock into position and supply the necessary support to the user. If patient experiences any skin irritation or severe discomfort contact your healthcare professional or medical technician.

If any serious incident or injury occurs please contact Therafin Corp and the competent authority for your country

BELTS, WEBBING, STRAPS AND UPHOLSTERED POSITIONING COMPONENTS.

Check all components regularly and replace if there are signs of tears, rips or fraying. Prevent foreign objects from entering locking buckles and insure they latch at each use.

Product is made of materials which can be disposed of safely without special precautions.

WARRANTY

Limited Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

Liability

While Therafin Corporation is responsible for any product that is defective due to manufacturing by Therafin Corporation, we are not liable for misuse or misapplication of our products. To achieve optimum performance, it is recommended that experienced professionals be consulted in selecting and applying our products.

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