Models:

50361

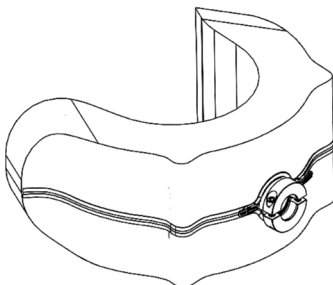
50363

50368

50369

2. **en** Provide posterior head support to address fatigue or to prevent extension.
6. **da** Giv bageste hovedstøtte for at tackle træthed eller for at forhindre forlængelse.
10. **nl** Bieden achterste hoofdsteen naar het adres van vermoeidheid of verlenging te voorkomen.
14. **fi** Tarjoa takaosan pään tuki väsymyksen korjaamiseksi tai venymisen estämiseksi.
18. **fr** Fournir un support de tête postérieur pour traiter la fatigue ou empêcher l'extension.
22. **de** Stellen Sie eine posteriore Kopfstütze bereit, um Ermüdung entgegenzuwirken oder eine Extension zu verhindern.
26. **it** Fornire un supporto per la testa posteriore per affrontare l'affaticamento o per prevenire l'estensione.
30. **no** Gi bakre hodestøtte for å adressere tretthet eller for å forhindre utvidelse.
34. **pt** Fornece suporte posterior para a cabeça para tratar a fadiga ou prevenir a extensão.
38. **es** Proporciono soporte posterior para la cabeza para abordar la fatiga o para evitar la extensión.
42. **sv** Ge bakre huvudstöd för att ta itu med trötthet eller för att förhindra förlängning.

Pro-Fit Head Support

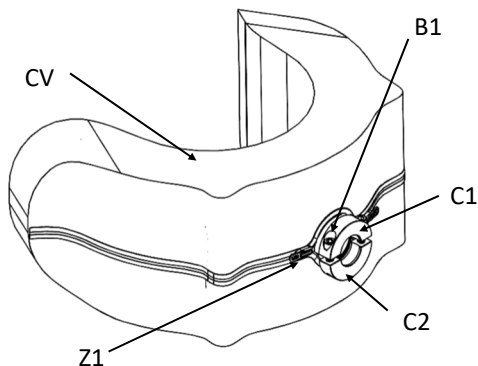
ENGLISH

en

User Instructions

-  Caution: Read Instructions before operating device.
-  Caution: Device should be installed by a trained wheelchair technician.
-  Caution: Device should be attached to a wheelchair back that conforms with ISO16840-3 or equivalent.
-  Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
-  Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.


Product Identification


Installation and Assembly

1: Open Cover (CV1) Loosen B1 using a 3/16 inch allen wrench.

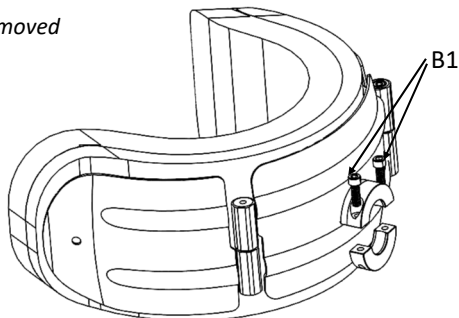**Shown with cover removed*

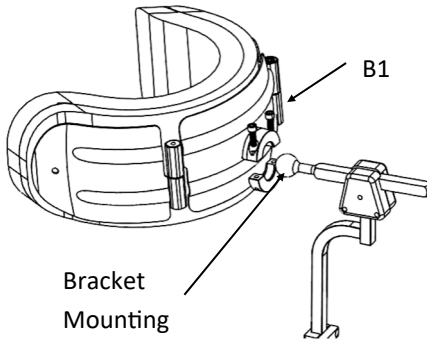
2: Insert Bracket Mounting Ball. Tighten B1. DO NOT PINCH COVER UNDER BALL.

To Adjust

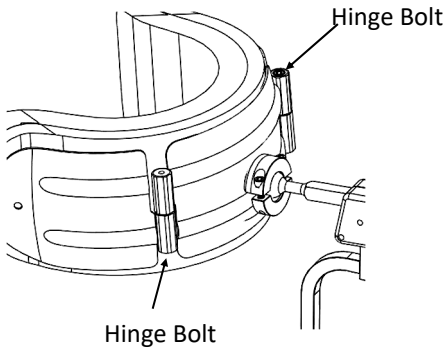
1: Loosen the Hinge Bolts with a 3/16 inch allen wrench. (Tap on the bolt if necessary to loosen the hinge joint.)

2: Adjust Side Panels to desired position and retighten bolts.

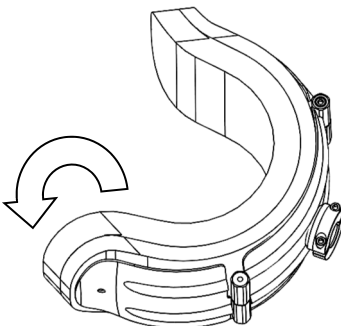Warning: Must tighten all bolts

Pro-Fit Head Support

User Instructions

Care and Cleaning

Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Remove cover for washing

Machine Wash in Warm Water

2. Check all fasteners and retighten as needed.

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

Notices

If patient experiences any skin irritation or severe discomfort contact your healthcare professional or medical technician.

If any serious incident or injury occurs please contact Therafin Corp and the competent authority for your country

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Kontakta Therafin Corp och den behöriga myndigheten i ditt land om någon allvarlig incident eller skada inträffar
MedEnvoy Global B.V.