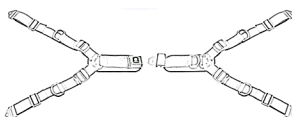
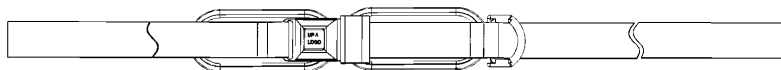
Models:


33130	60126	60173	60433	60492	61203	30517TOP	60447TOP
60024	60127	60183	60443	60493	62003	30518TOP	60448TOP
60025	60128	60310	60453	60494	62103	60221TOP	60449TOP
60026	60130	60313	60463	60495	62203	60241TOP	60450TOP
60046	60131	60323	60473	60496	64003	60437TOP	60457TOP
60047	60132	60333	60483	60497	64103	60438TOP	
60048	60133	60413	60490	61003	64303	60439TOP	
60125	60163	60423	60491	61103	30516TOP	60440TOP	

2. (en) To provide anterior pelvic support for those utilizing a wheelchair
5. (nl) Voorste bekkenondersteuning bieden aan mensen die een rolstoel gebruiken
8. (fr) Pour fournir un soutien pelvien antérieur pour ceux qui utilisent un fauteuil roulant
11. (de) Zur Unterstützung des vorderen Beckens für Rollstuhlfahrer
14. (it) Per fornire supporto pelvico anteriore a coloro che utilizzano una sedia a rotelle
17. (pt) Para fornecer suporte pélvico anterior para aqueles que utilizam uma cadeira de rodas
20. (es) Para proporcionar soporte pélvico anterior para quienes utilizan una silla de ruedas.

-  Caution: Read Instructions before operating device.
-  Caution: Device should be attached to a wheelchair seat that conforms with ISO16840-3 or equivalent.
-  Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
-  Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.
-  Caution: Device should be installed by a trained wheelchair technician.
-  Warning: This device is NOT intended as an occupant restraint in a moving vehicle. This device is for positioning use only and should not be relied upon as a transit safety restraint. Use only WC-3 tested products for occupant restraint in a moving vehicle.

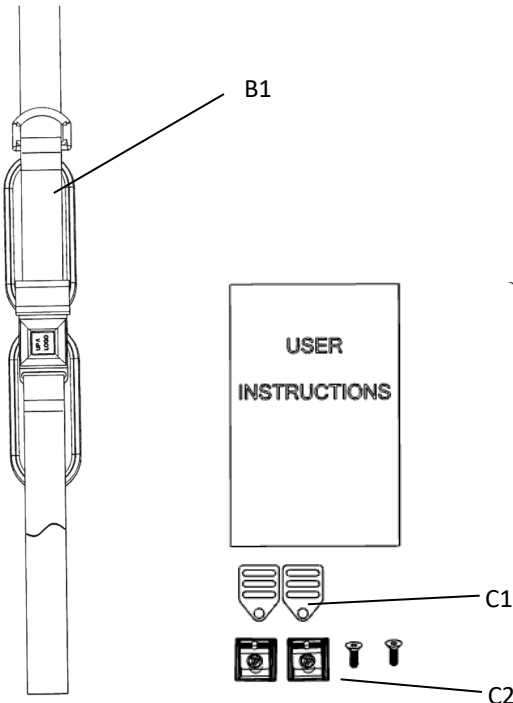
Product Identification

To Install:


FOR C1:

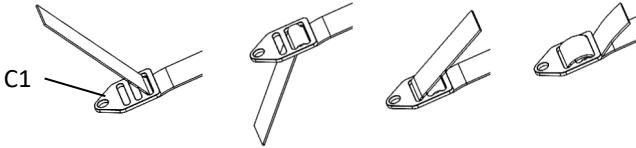Step 2: Feed straps through C1 as shown.


FOR C2:

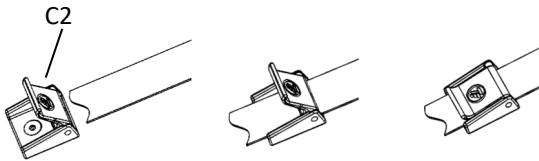
Step 2: Feed straps through C2 as shown.

To Adjust:

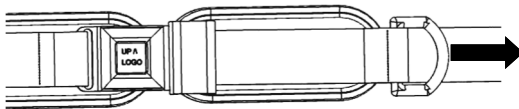To Release:

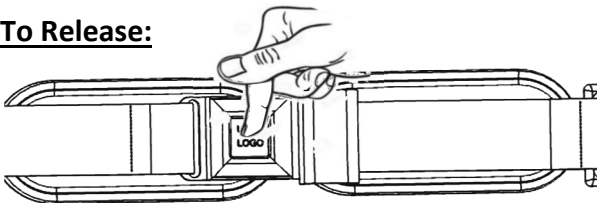Care and Cleaning

 Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth.

2. Check all fasteners and retighten as needed.

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

Notices

If patient experiences any skin irritation or severe discomfort contact your healthcare professional or medical technician.

If any serious incident or injury occurs please contact Therafin Corp and the competent authority for your country.

Product is made of materials which can be disposed of safely without special precautions.

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Si ocurre algún incidente o lesión grave, comuníquese con Therafin Corp y la autoridad competente de su país.
MedEnvoy Global B.V.