Models:


30516TOP
30517TOP
30518TOP
30434
30435

2. **en** To provide anterior pelvic support for those utilizing a wheelchair
5. **nl** Voorste bekkenondersteuning bieden aan mensen die een rolstoel gebruiken
8. **fr** Pour fournir un soutien pelvien antérieur pour ceux qui utilisent un fauteuil roulant
11. **de** Zur Unterstützung des vorderen Beckens für Rollstuhlfahrer
14. **it** Per fornire supporto pelvico anteriore a coloro che utilizzano una sedia a rotelle
17. **pt** Para fornecer suporte pélvico anterior para aqueles que utilizam uma cadeira de rodas
20. **es** Para proporcionar soporte pélvico anterior para quienes utilizan una silla de ruedas.


-  Caution: Read Instructions before operating device.
-  Caution: Device should be attached to a wheelchair seat that conforms with ISO16840-3 or equivalent.
-  Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
-  Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.
-  Caution: Device should be installed by a trained wheelchair technician.
-  Warning: This device is NOT intended as an occupant restraint in a moving vehicle. This device is for positioning use only and should not be relied upon as a transit safety restraint. Use only WC-3 tested products for occupant restraint in a moving vehicle.

Product Identification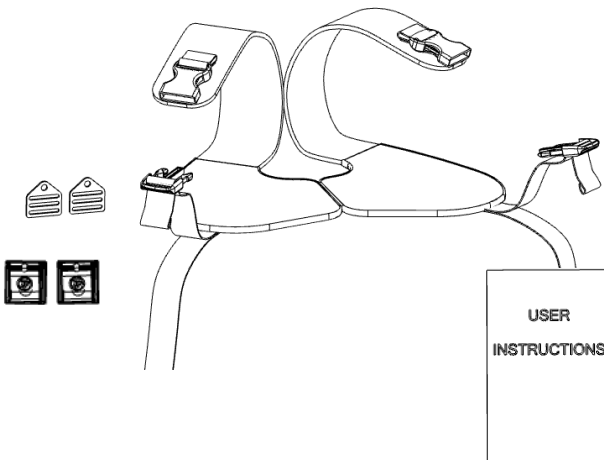


To Install:


1. Mount attaching clips to chair as desired and thread strap as shown.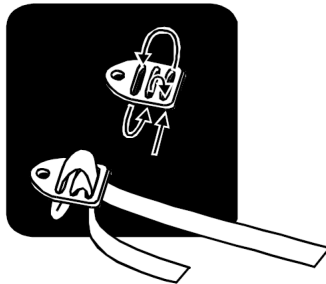

2. Place harness in chair as shown below and wrap pads around user's legs.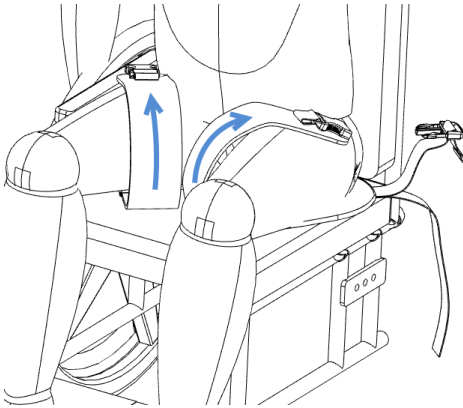

3. Fasten buckles around either leg of the user and pull strap to tighten.

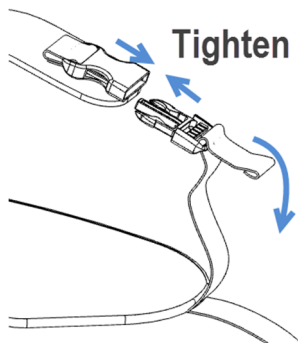Care and Cleaning

 Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth.

2. Check all fasteners and retighten as needed.

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

Notices

If patient experiences any skin irritation or severe discomfort contact your healthcare professional or medical technician.

If any serious incident or injury occurs please contact Therafin Corp and the competent authority for your country.

Product is made of materials which can be disposed of safely without special precautions.

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Si ocurre algún incidente o lesión grave, comuníquese con Therafin Corp y la autoridad competente de su país.
MedEnvoy Global B.V.