**en**

To provide upper extremity support for a wheelchair user.

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Caution: Read Instructions before operating device.

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Caution: Device should be installed by a trained wheelchair technician.

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Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.

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Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

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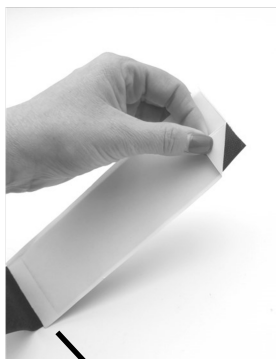
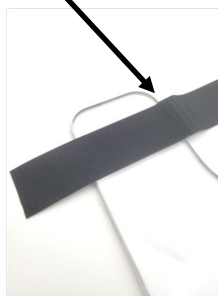
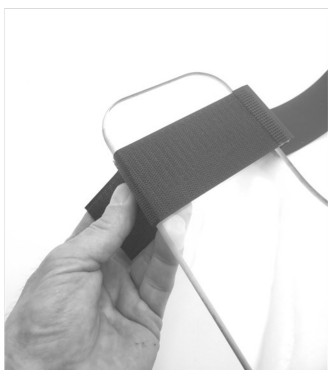
3

2X

If patient experiences any skin irritation or severe discomfort contact your healthcare professional or medical technician. If any serious incident or injury occurs please contact Therafin Corp. and the competent authority for your country. Product is made of materials which can be disposed of safely without special precautions.

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Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corp.. This replacement warranty does not include misuse or neglect of the product.

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MedEnvoy Global B.V.
Therafin Corporation