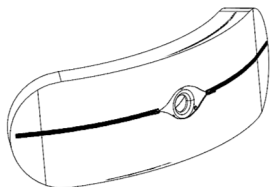
Pro-Fit Soft Pad Head Support

<u>Models:</u>	<u>Covers:</u>
	50711
50365	50712
50374	50713
	50714
50366	50715
50375	50716
	50346
50367	50347
	50348
50376	50351
	50352
	50353

2. (en) Provide posterior head support to address fatigue or to prevent extension.
6. (da) Giv bageste hovedstøtte for at tackle træthed eller for at forhindre forlængelse.
10. (nl) Zorg voor achterzijde hoofdsteun om vermoeidheid tegen te gaan of om verlenging te voorkomen.
14. (fi) Tarjoa takaosan pään tuki väsymyksen korjaamiseksi tai venymisen estämiseksi.
18. (fr) Fournir un support de tête postérieur pour traiter la fatigue ou empêcher l'extension.
22. (de) Stellen Sie eine posteriore Kopfstütze bereit, um Ermüdung entgegenzuwirken oder eine Extension zu verhindern.
26. (it) Fornire un supporto per la testa posteriore per affrontare l'affaticamento o per prevenire l'estensione.
30. (no) Gi bakre hodestøtte for å adressere tretthet eller for å forhindre utvidelse.
34. (pt) Fornece suporte posterior para a cabeça para tratar a fadiga ou prevenir a extensão.
38. (es) Proporciona soporte posterior para la cabeza para abordar la fatiga o para evitar la extensión.
42. (sv) Ge bakre huvudstöd för att ta itu med trötthet eller för att förhindra förlängning.

Soft Headrest Pad

English

-  Caution: Read Instructions before operating device.
-  Caution: Device should be installed by a trained wheelchair
-  Caution: Device should be attached to a wheelchair back that conforms with ISO16840-3 or equivalent.
-  Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
-  Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.
-  Caution: Use caution near ignition sources or open flame.

Product Identification

Small, with Smooth Cover

Small, with Knit Cover

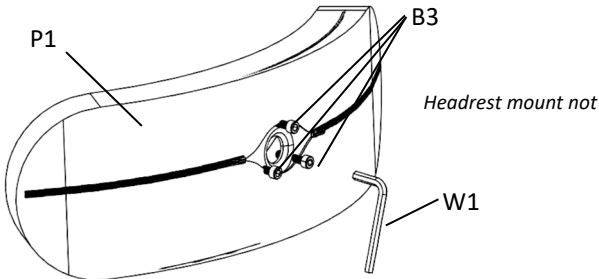
Medium, with Smooth Cover

Medium, with Knit Cover

Large, with Smooth Cover


Large, with Knit Cover

Components of the Soft Headrest Pad.


Installation and Assembly

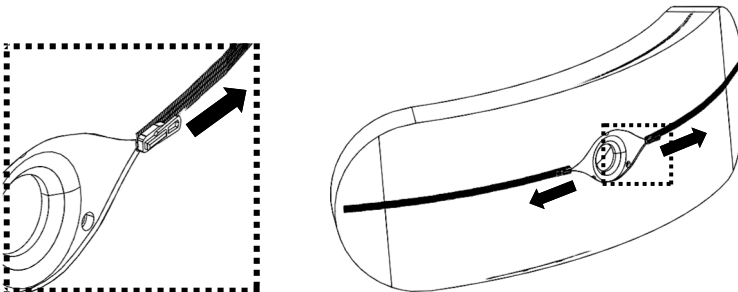1. Unzip Cover



2. Align **Headrest Mount** with holes in P1, install B3.

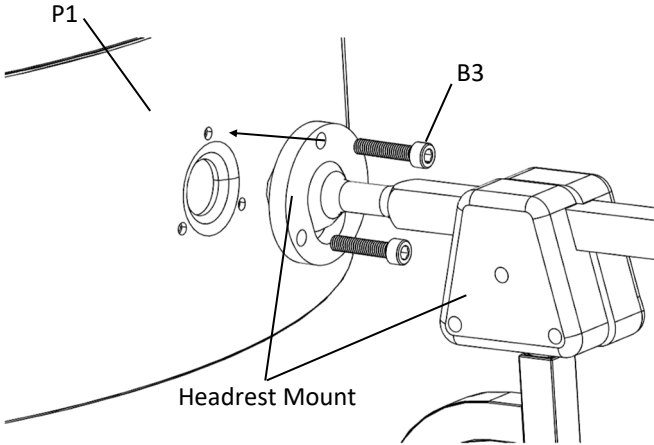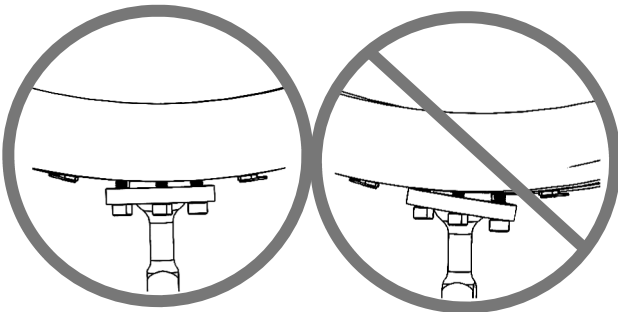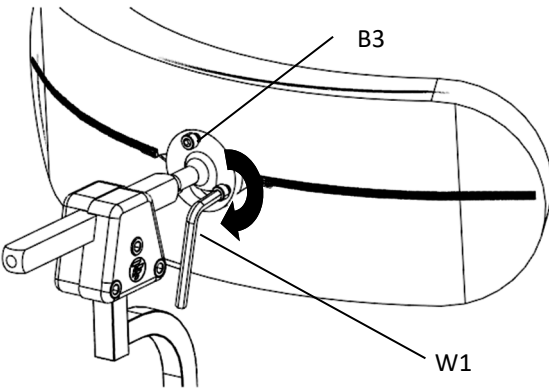(**Headrest Bracket** not included See separate instructions to install **Bracket**.)

3. Tighten each B3 same number of revolutions using W1. Must tighten securely.

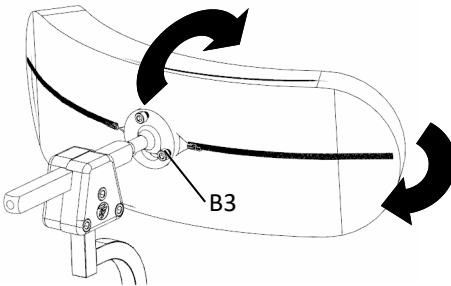
Soft Headrest Pad

Adjustment

-  Caution: Device should be installed by a trained wheelchair
-  Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
-  Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

Loosen Fasteners B3. Make adjustments shown below. Re-tighten all fasteners after desired position is obtained.

Notices

If patient experiences any skin irritation or severe discomfort contact your healthcare professional or medical technician.

If any serious incident or injury occurs please contact Therafin Corp and the competent authority for your country.

Product is made of materials which can be disposed of safely without special precautions.

Care and Cleaning

 Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth.

2. Check all fasteners and retighten as needed.

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

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MEDENVOY GLOBAL B.V.