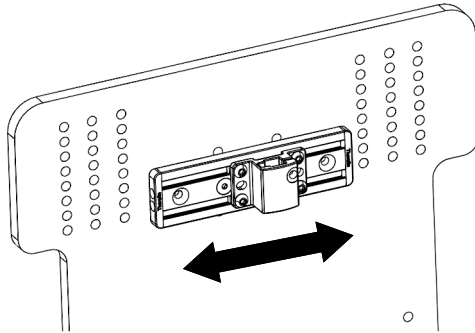
Ajustar:

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
Headrest Horizontal Slide Mount

English


Image below shows installed device.

32975

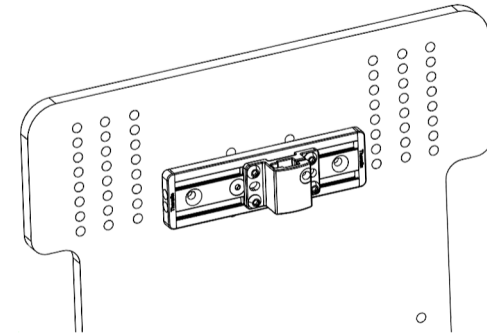Use: To Provide posterior head support.


Product Identification

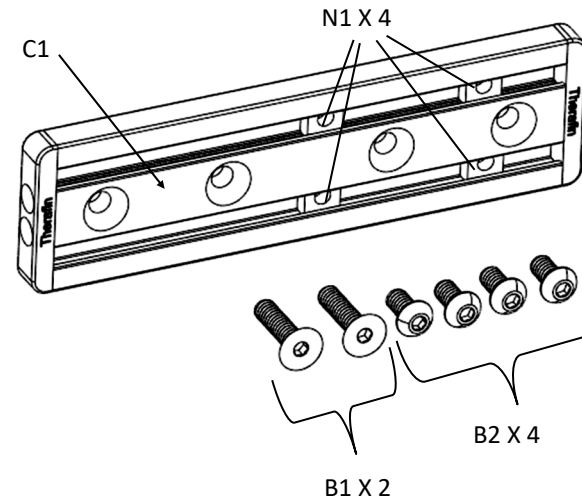
- ⚠ Caution: Read Instructions before operating device.
- ⚠ Caution: Device should be installed by a trained wheelchair technician.
- ⚠ Caution: Device should be attached to a wheelchair back that conforms with ISO16840-3 or equivalent.
- ⚠ Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
- ⚠ Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

To Inst all:

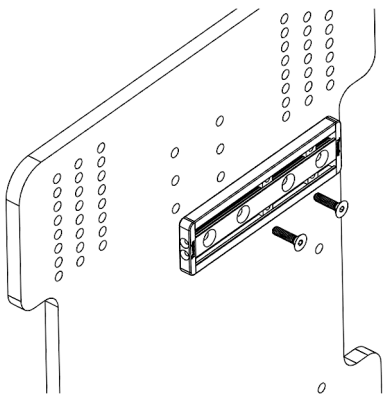Step 1: Align C1 with holes in backrest. Install using bolts B1 and tighten securely.

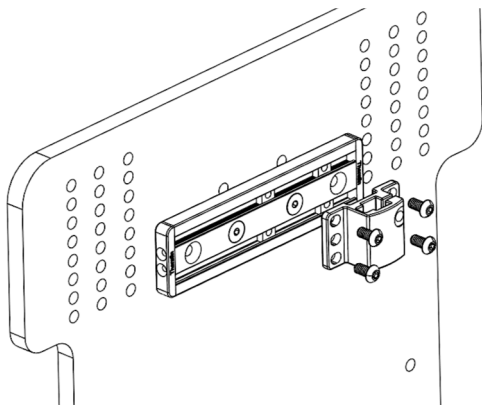
Step 2: Attach Headrest Clamp (not included) using bolts B2

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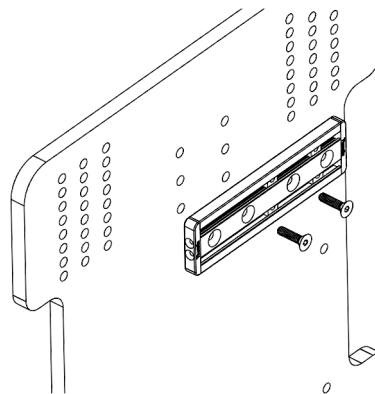
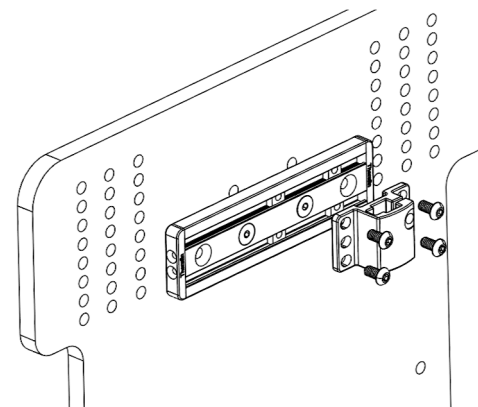
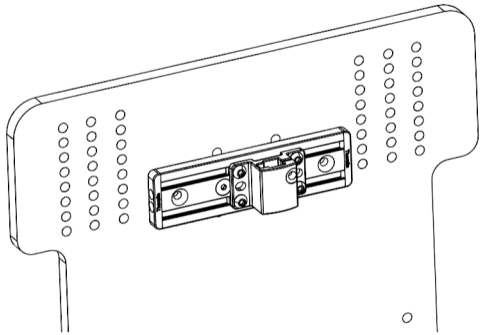
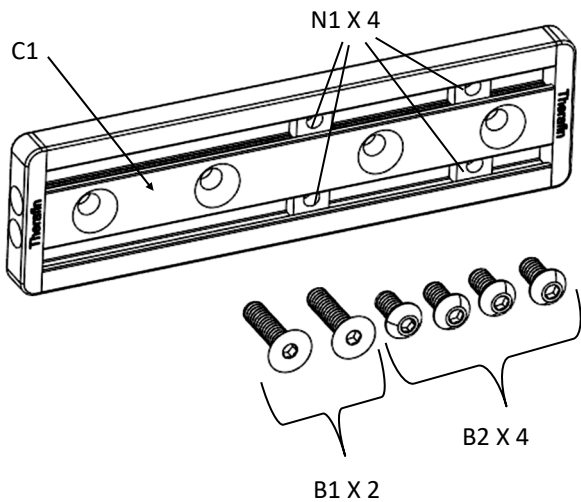
To Adjust:

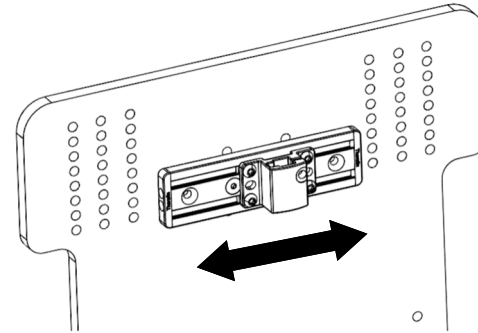Loosen Bolts B2 and slide clamp along channel C1.

Care and Cleaning

Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth.

2. Check all fasteners and retighten as needed.

Notices

If patient experiences any skin irritation or severe discomfort contact your healthcare professional or medical technician.

If any serious incident or injury occurs please contact Therafin Corp and the competent authority for your country

Product is made of materials which can be disposed of safely without special precautions.

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

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En cas d'incident ou de blessure grave, veuillez contacter Therafin Corp et l'autorité compétente de votre pays
MedEnvoy Global B.V.