Models:


30594	30597	30604	30626
30589	30592	30618	30621
30595	30598	30624	30627
30590	30593	30619	30622
30596	30603	30625	
30591	30601	30620	

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

TheraFit Vest Extended Straps

TheraFit Vest Comfort Fit Straps

en English Image below shows installed device.

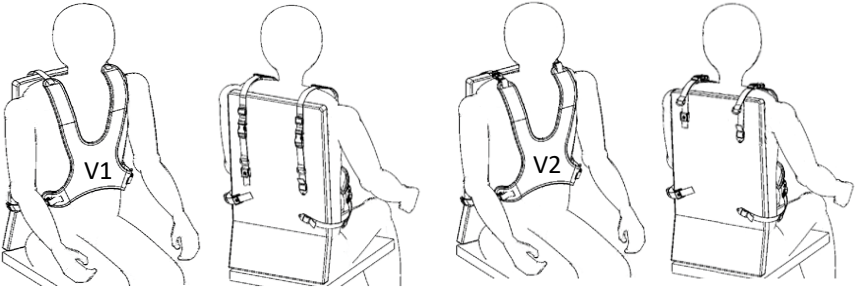Product Identification

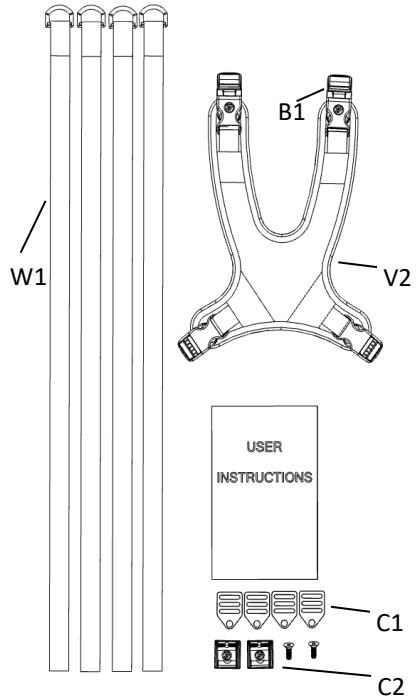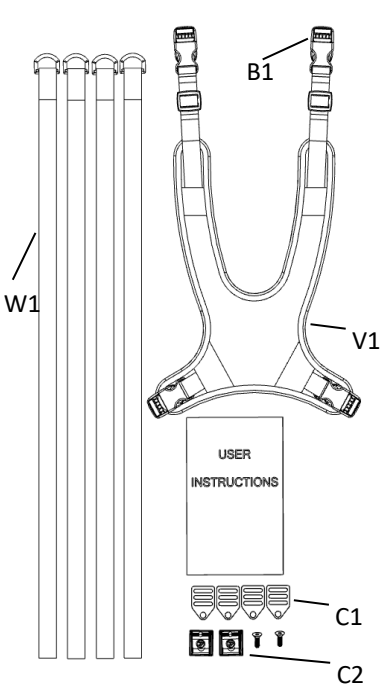Caution: Device should be installed by a trained wheelchair technician.

Caution: Device should be attached to a wheelchair back that conforms with ISO16840-3 or equivalent.

Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.

Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

Warning: This device is NOT intended as an occupant restraint in a moving vehicle. This device is for positioning use only and should not be relied upon as a transit safety restraint. Use only WC-3 tested products for occupant restraint in a moving vehicle.

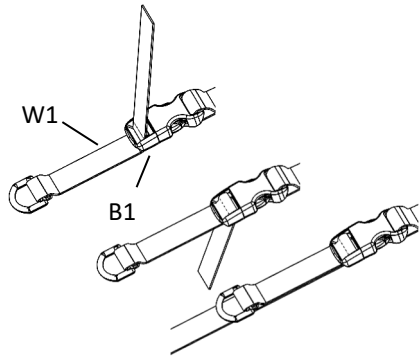
Warning: This device should be worn properly. Position the device properly to avoid choking hazard. Device should always be used with a hip positioning belt.


To Install:

Step 1: Select attaching method:

2 of Clips (C1) and 2 of Cams (C2) , or
4 Clips (C1)

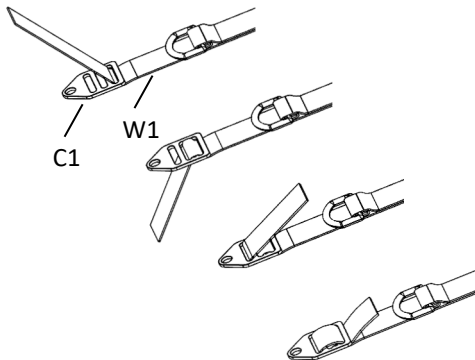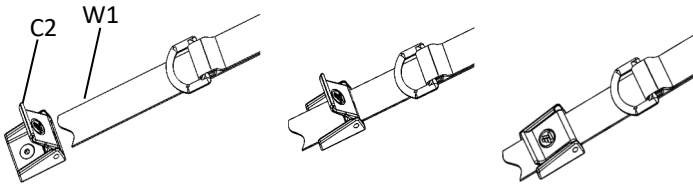
Step 2: Feed webbing (W1) through buckles (B1) as shown.

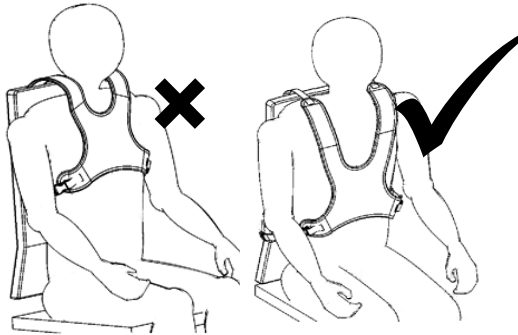
 Caution: Read Instructions before operating device.

Step 3: Feed webbing (W1) through clips (C1) and cams (C2) as shown.

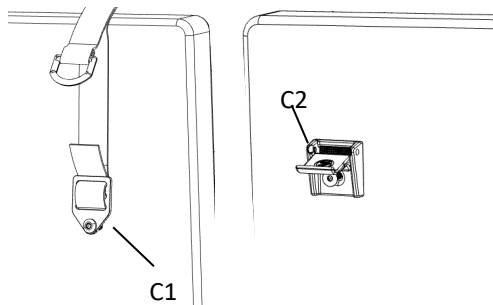
Step 4: Position device on patient. Do not place too high near neck.

 Warning: This device should be worn properly. Position the device properly to avoid choking hazard. Device should always be used with a hip positioning belt.


Step 5: Attach clips (C1) and cams (C2) to wheelchair back. Tighten fasteners. (Fasteners to secure C1 *NOT INCLUDED*)

To Adjust:

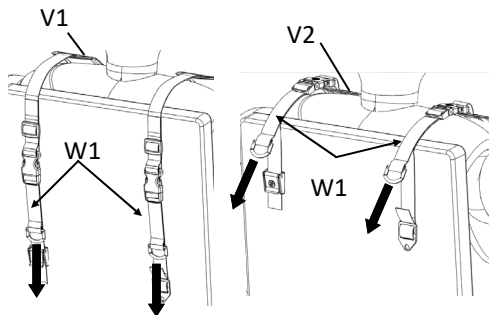Once all clips (C1) and cams (C2) are secured, tighten straps (W1) by pulling on D-Ring. Be sure vest is NOT positioned too high near neck.

Care and Cleaning

Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth. Air Dry.

2. Check all fasteners and retighten as needed.

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

Document 08662

Revision E 6-2022

If patient experiences any skin irritation or severe discomfort contact your healthcare professional or medical technician.

If any serious incident or injury occurs please contact Therafin Corp and the competent authority for your country.

Product is made of materials which can be disposed of safely without special precautions.

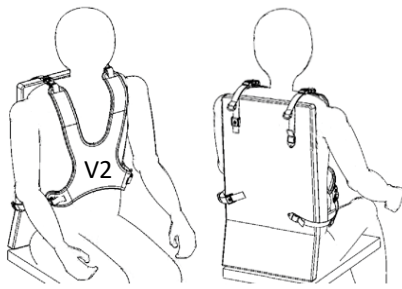
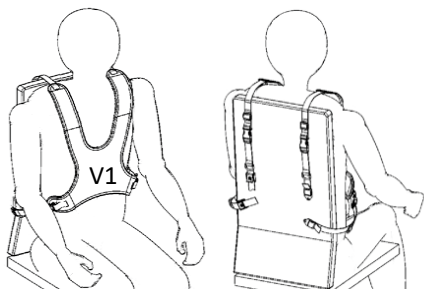
Therafin Corporation
9450 W Laraway Road
Frankfort, IL 60423 USA
800-843-7234
www.therafin.com

MedEnvoy Global B.V.
Prinses Margrietplantsoen 33
Suite 123
2595 AM The Hague
The Netherlands



TheraFit Vest Udvidede Stropper

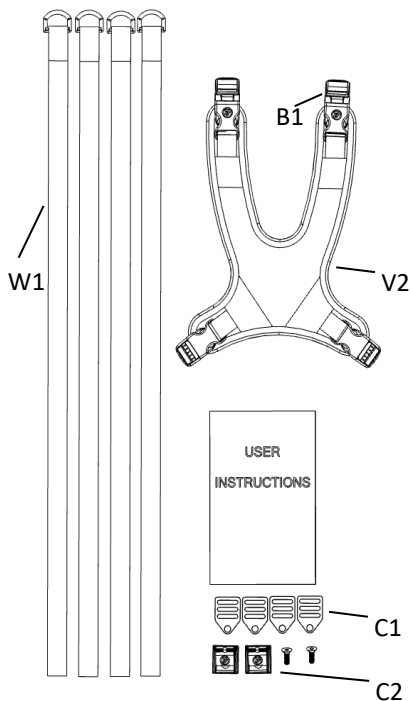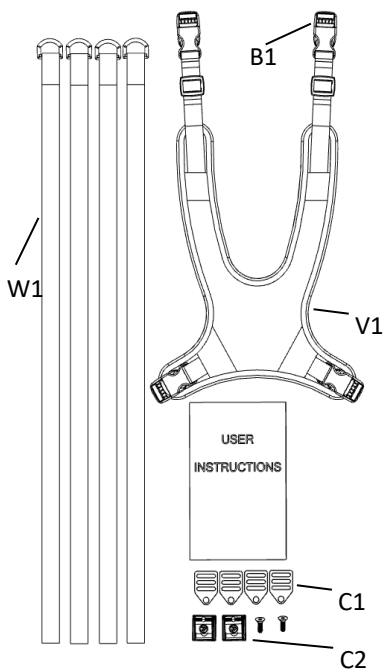TheraFit Vest Comfort Fit Stropper

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