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
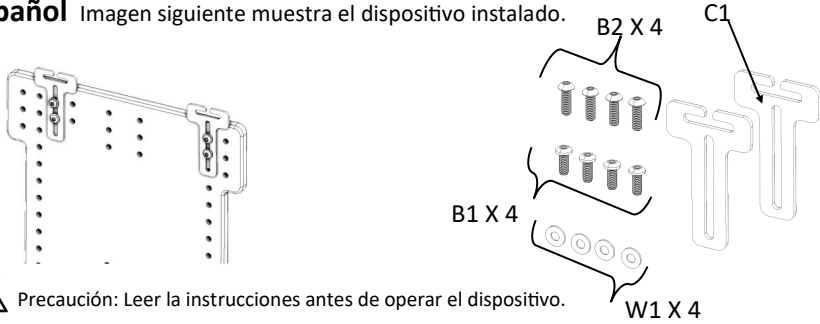
51030 Use: To provide support to the torso of a wheelchair user to promote upright posture.

English Image below shows installed device.

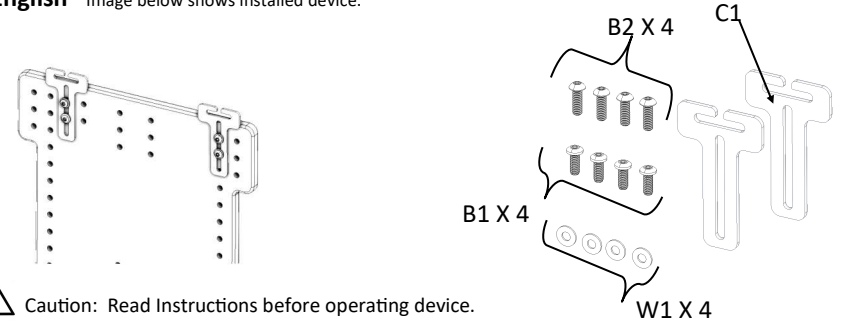
Product Identification

- ⚠ Caution: Read Instructions before operating device.
- ⚠ Caution: Device should be installed by a trained wheelchair technician.
- ⚠ Caution: Device should be attached to a wheelchair back that conforms with ISO16840-3 or equivalent.
- ⚠ Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
- ⚠ Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

To Install:

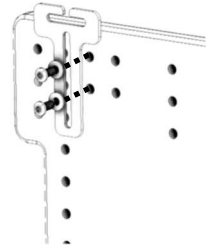Install on Back as shown. Use Longer screws if stacking on top of Back Mounting Hardware.

Care and Cleaning

- ⚠ Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth.

2. Check all fasteners and retighten as needed.

Notices

If patient experiences any skin irritation or severe discomfort contact your healthcare professional or medical technician.

If any serious incident or injury occurs please contact Performance Health and the competent authority for your country
Product is made of materials which can be disposed of safely without special precautions.

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Performance Health. This replacement warranty does not include misuse or neglect of the product.

MedEnvoy Global B.V.
Prinses Margrietplantsoen 33- Suite 123
2595 AM The Hague
The Netherlands

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Therafin Corporation